Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

03:2019-02
300030282

EN

**READ CAREFULLY
BEFORE USE**

AcryPoints are used for adjusting, finishing and polishing acrylic dentures or other acrylic appliances as well as plastic impression trays. Use AcryPoints "Coarse" (dark grey) for adjustment, "Medium" (brown) for finishing and "Fine" (light grey) for polishing.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

STERILIZATION

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization.
- When necessary, remove any contaminants with cleaning liquid or disinfectant, then sterilize.
- Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion.
- Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

MAXIMUM PERMISSIBLE ROTATION SPEED

AcryPoint	15,000 min ⁻¹
-----------	--------------------------

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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