SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

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en READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

Ceramisté silicone polishers are for polishing porcelain and enamel. Ceramisté Standard polishers are for prepolishing, Ceramisté Ultra polishers (yellow band) are for polishing and Ceramisté Ultra II polishers (white band) are for superpolishing.

PRECAUTIONS AND WARNINGS

●Never exceed the maximum permissible rotation speeds described in Table A. ●Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points. ●Mount an unmounted wheel onto the mandrel securely. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Always use an unmounted wheel in clockwise motion. ●Use local dust extractor and protective equipment while using. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use. ●Dispose in accordance with relevant local and national regulation.

STORAGE AND STERILIZATION

●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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