de Gebrauchsanleitung
es Instrucciones de uso
fr Instructions per l'uso
it Istruzioni per l'uso
pl Instrukcja obsługi
pt Instruções de uso

en Instructions for use
fr Mode d'emploi
it Istruzioni per l'uso
pl Instrukcja obsługi
pt Instruções de uso

04.10.2020-09
71296

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Atrium, Singapore 117664

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92076-4095, USA

en

READ CAREFULLY BEFORE USE

Bonding agent for porcelain and resin

CERARESIN BOND is a light-curing bonding system for repairs of fractured dental ceramic or resin restorations (prostheses) using resin matrix.
Consisting of two articles, BOND I and BOND II, CERARESIN BOND is intended for intraoral and extraoral repairs of restorations (prostheses) in conjunction with C&B composite resin, direct filling composite resin or acrylic resin.

COMPOSITION
BOND I: Ethanol, Silane coupling agent and others
BOND II: Acetone, 4-AET, UDMA, Polymerization initiator and others

INDICATIONS
CERARESIN BOND is used as adhesive for:
➢ repairs of fractures in ceramic, direct or indirect composite resin restorations (prostheses) using direct or indirect composite resin (eg, repairs of facing crowns, veneers and implant superstructures)
➢ repairs of fractures in artificial teeth (porcelain, hard resin or acrylic resin teeth) using direct or indirect composite resin
* Use M.L. Primer (sold separately) when the repair includes metal surface.

- (eg, repairs of metal-keramic restorations)
- PRECAUTIONS**
- Do not use this product on patients with known allergies to this material and/or methacrylate monomer.
 - Operators with known allergies to this material and/or methacrylate monomer must not use this product.
 - If any inflammation or other allergic reactions occur on either the patient or operator while using this product, immediately discontinue use and seek medical advice.
 - This product contains ethanol and acetone. If inhalation of vapor of these materials causes nausea or illness, take rest and keep warm in fresh air.
 - Do not handle the liquid or the mixture of this material with bare hands. Use protective plastic gloves and glasses to avoid contact with skin and/or eyes. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of running water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
 - This product is intended for use by dental professionals only.

- DIRECTIONS FOR USE**
- Treatment of the restoration surface (pretreatment)
For intraoral repair
Roughen the restoration surface with abrasives (Diamond point etc.) Rinse with water and dry. Then, treat the surface with phosphoric acid based etching agent if necessary. Rinse with water and dry.
For extraoral repair
Roughen the restoration surface with abrasives (Diamond point etc.) or sandblast with 50 – 100 µm alumina particles. Rinse with water. Then, treat the surface with phosphoric acid based etching agent if necessary. Rinse with water and dry.
 - For repairing metal-ceramic restoration including metal surface
2-1 Application of M.L. Primer (sold separately)
Apply one coat of M.L. Primer using brush and leave undisturbed for 10 sec.
2-2 Application of opaquer (sold separately)
Apply one layer of opaquer on the conditioned metal surface and light-cure using a dental handheld or laboratory light-curing unit.
 - Application and drying of BOND I
Apply one coat of BOND I on the entire surface to be repaired using a brush. Leave it undisturbed for 10 sec.
4. Application and drying of BOND II
Apply one coat of BOND II on the entire surface to be repaired using a brush. Leave it undisturbed for 10 sec.
5. Light-curing (irradiation)
Light-cure the surface with dental handheld or laboratory light-curing unit.

Light-curing unit	Standard curing time	Performance of light-curing unit
Halogen	Labo- ratory 3 min.	wavelength: 400-500 nm, light intensity: ≥ 500 mW/cm ²
	Hand- held 20 sec.	wavelength: 440-490 nm, light intensity: ≥ 1,000 mW/cm ²
LED	10 sec.	wavelength: 440-490 nm, light intensity: ≥ 1,000 mW/cm ²

6. Application of composite resin
Follow the instructions for use of resin material to be used.

- NOTE ON USE**
- Tightly close the bottle cap immediately after each dispensing as it contains volatile agents. Use all the dispensed material promptly.
 - Use caution to avoid mixing BOND I and BOND II in the dish. Use separate brushes (MICROBRUSH) for respective materials.
 - For intraoral repairs, do not reuse the brush to avoid cross-contamination.
 - Use all the dispensed material on one patient.
 - Operators must use protective glasses etc to avoid looking directly at the curing light.
 - Provide any source of ignition since this product is flammable.
 - Allow the material to reach room temperature before use when storing this product in the refrigerator.
 - Use in well ventilated area with several appropriate ventilations per hour.
 - Keep a fire extinguisher in the area where this product is used or stored.

10. Follow the instructions for use of the dental materials or instruments to be used in conjunction with this product.
 11. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
 12. Use the expiration date indicated on the package and container. (Example ☞ YYYY-MM-DD → Year-Month-Date of the expiration date)
- STORAGE**
1. Store at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep out of direct sunlight and any source of ignition.
 2. Do not store this product in large amount in one single storage container or storage place.

CAUTION: US Federal law restricts this product to use by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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