SpruCut/Dura-Green CHIPLESS WHEELS

English Instructions for Use

Chipless Wheels (silicone carbide grit) are heatless wheels for fine finishing of porcelain and precious alloys.
SpruCut Wheels (aluminium oxide grits) are for gross reduction of cobalt-chrome and nickel alloys, i.e. trimming of cut off sprues.

	Recommended r.p.m.	Maximum r.p.m.	Recommended pressure
Chipless Wheels	5,000-12,000	20,000	2 N
SpruCut Wheels	5,000-15,000	20,000	3 N

Caution

- * Operate each instrument up to maximum permissible speed only.
- * Mount the instruments onto the mandrel tightly and insert the shank completely into the chuck of handpiece.
- * Rotate each instrument each time prior to use to check smooth run and safety.
- * Operate at recommended pressure only.
- * Do not use at an acute angle.
- * Wear eye protecting glasses when using.
- * Use these products for indicated purpose only.

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Manufacturer
SHOFU INC, 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0963, Japan
SHOFU DENTAL ASIA-PACIFIC PTE. LTD., 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
SHOFU DENTAL CORPORATION, 1225 Stone Drive, San Marcos, California 92078-4059, USA
SHOFU UK, Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD., No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
SHOFU DENTAL GmbH, Am Brühl 17, 40878 Ratingen, Germany