## PRODUCT DESCRIPTION

CompoMaster are impregnated with diamond grains in a synthetic rubber binder. CompoMaster Coarse are intended for finishing and polishing of composite restorations and CompoMaster is for super-polishing them. For contouring, Dura-Green Stones or Dura-Green Stones Fine is recommended. For further super-polishing, DirectDia paste or SuperBuff polishing discs is recommended.

## PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds.
- Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion.
- Use local dust extractor and protective equipment while using.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.
- Dispose in accordance with relevant local and national regulation.

## STORAGE AND STERILIZATION

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- Perform autoclave under the condition described in Table B.
- When using intraorally, sterilize before each use.
- For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.
- Follow the instructions for use of washer disinfectant and sterilizer, respectively.
- Do not use chlorine disinfectant liquid.
- For the details about REPROCESSING procedures, visit www.shofu.de.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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