**SHOFU DENTAL CORPORATION**  
 1225 Stone Drive, San Marcos,  
 California 92078-4059, USA

**SHOFU DENTAL ASIA-PACIFIC PTE.LTD.**  
 10 Science Park Road, #03-12,  
 The Alpha, Science Park II, Singapore 117684

**EC REP**

**SHOFU DENTAL GmbH**  
 An der Pönt 70, 40885 Ratingen, Germany

04.0:2020-09  
 300030338

 **CompoSite**  
**CompoSite Fine**

**en**

**READ CAREFULLY BEFORE USE**

**PRODUCT DESCRIPTION**

CompoSite and CompoSite Fine polishers will polish and final polish composite restorations without scratching surrounding tooth structure. Use CompoSite for finishing and pre-polishing and CompoSite Fine for polishing.

**PRECAUTIONS AND WARNINGS**

- Never exceed the maximum permissible rotation speeds described in Table A. ● Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points. ● Insert the shank completely into the chuck of the handpiece. ● Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ● Perform test run to check safety and smooth operation without run-out prior to use. ● Avoid any use at acute angles or with excessive pressure. ● Use local dust extractor and protective equipment while using. ● This product is intended for use by dental professionals only. ● Do not use this product for any purposes other than the intended use. ● Dispose in accordance with relevant local and national regulation.

**STORAGE AND STERILIZATION**

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ● Perform autoclave under the condition described in Table B. ● When using intraorally, sterilize before each use. ● For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ● Follow the instructions for use of washer disinfectant and sterilizer, respectively. ● Do not use chlorine disinfectant liquid. ● For the details about REPROCESSING procedures, visit www.shofu.de.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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