SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

**EC REP**

SHOFU DENTAL GmbH  
An der Pönt 70, 40885 Ratingen, Germany

04.0:2020-09  
72432

 **Composite Finishing Kit**

**en** **READ CAREFULLY BEFORE USE**

**PRODUCT DESCRIPTION**

This kit is designed for reducing and finishing composite fillings, also interproximal and subgingival. It has six Dura-Green and six Dura-White Stones in CN1 and FL2 shapes. Dura-Green Stones will fast finish composite restorations. Dura-White Stones will fine finish composite restorations. Dura-Green and Dura-White Stones are autoclavable for reuse.

**PRECAUTIONS AND WARNINGS**

- Never exceed the maximum permissible rotation speeds described in Table A.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.
- Dispose in accordance with relevant local and national regulation.

**STORAGE AND STERILIZATION**

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- Perform autoclave under the condition described in Table B.
- When using intraorally, sterilize before each use.
- For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.
- Follow the instructions for use of washer disinfectant and sterilizer, respectively.
- Do not use chlorine disinfectant liquid.
- Do not perform plasma sterilization.
- For the details about REPROCESSING procedures, visit [www.shofu.de](http://www.shofu.de).

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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