Composite Polishing Kit

04.0:2020-09
72433

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

CE 0123

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Dura-Green Stones	30,000 min ⁻¹
Dura-White Stones	30,000 min ⁻¹
CompoSite	20,000 min ⁻¹
CompoSite Fine	20,000 min ⁻¹

en MAXIMUM PERMISSIBLE ROTATION SPEED
de MAXIMAL ZULÄSSIGE DREHZAHL
fr VITESSE MAXIMALE DE ROTATION AUTORISÉE
nl MAXIMAAI TOELAATBARE ROTATIESNELHEID
es VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN
it MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
sv HÖGSTA TILLÅTNA ROTATIONSASTIGHET
pt VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

en

READ CAREFULLY BEFORE USE

de

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nl

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PRODUCT DESCRIPTION

This kit is designed for finishing and polishing all types of composite restorations. It includes two Dura-Green and two Dura-White Stones for finishing, six CompoSite for prepolishing and polishing and two CompoSite Fine (white band) Polishers for superpolishing. Dura-Green and Dura-White Stones, CompoSite and CompoSite Fine Polishers are autoclavable for reuse.

PRECAUTIONS AND WARNINGS

●Never exceed the maximum permissible rotation speeds described in Table A. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use. ●Dispose in accordance with relevant local and national regulation.

STORAGE AND STERILIZATION

●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●Dura-Green Stones and Dura-White Stones: Do not perform plasma sterilization. ●For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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