Dura-Green DIA

en Instructions for use

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1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

en

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

Dura-Green DIA is a diamond impregnated abrasive with glass binder. SHOFU ADJUSTER is available as an accessory to dress Dura-Green DIA.

INTENDED PURPOSE

Contouring and finishing of indirect restorations

- Porcelain
- Zirconia
- Lithium Disilicate etc.

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotating speed.

30,000 min⁻¹

- Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
- Use protective equipment such as glasses while using.
- Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
- Use SHOFU ADJUSTER to remove clogging of the debris or to adjust the abrasive to the desired size.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.

STORAGE AND STERILIZATION

Store this product avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. When using intraorally, sterilize this product before each use. For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.

(Autoclave)

Cycle Type	Temperature	Holding time	Dry time
Pre-vacuum 3 times	134°C 273°F	3'00"	10'00"

Follow the instructions for use of washer disinfectant and sterilizer, respectively. Chlorine disinfectant must not be used to avoid corrosion of the instruments. For the details about REPROCESSING procedures, visit www.shofu.de.

DISPOSAL

Dispose in accordance with relevant local and national regulation.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

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