



ENDURA

EN Instructions for use

02:2015-02
W000002



Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EN HIGH WEAR RESISTANCE PLASTIC TEETH

ENDURA are three-layer hybrid resin teeth that provide excellent wear resistance.

SHADES

Anterior (15 shades):

102, 104, 106, 108, 109, 110, 112, 114, 116, A1, A2, A3,
A3.5, B2, B3

Posterior (8 shades):

65, 66, 67, 70, A2, A3, A3.5, B3

ZERO posterior (2 shades):

A3, A3.5

COMPOSITION

Anterior

Enamel: UDMA, Micro fumed silica, others

Dentin: PMMA, EDMA, others

Base: PMMA, others

Posterior/ZERO posterior

Enamel: UDMA, Micro fumed silica, others

Dentin: UDMA, Micro fumed silica, others

Base: PMMA, others

INDICATIONS

Used as denture teeth to produce partial or full dentures

PRECAUTIONS

1. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice
2. Wear protective glasses etc. while cutting and polishing this product to avoid damages to eyes.
3. Use a dust extractor, dust protective mask etc. while cutting or polishing this product, to avoid harmful effects of dust on the human body.
4. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
5. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Produce full or partial dentures following the conventional method.

- Carefully contour and polish this product since its enamel and dentin layers are hard and highly wear resistant.
- Do not use strong alkaline gypsum dissolving agent (pH 10 or over) to avoid discoloration.
- Do not expose to fire when fabricating wax denture.
- Give mechanical retention when using thermoplastic denture base resins. ENDURA does not bond to thermoplastic denture base resins.

- Give the following general instructions to patient:

- a. Keep the denture clean. Gently remove food debris with a soft denture brush after eating. Using denture cleanser is more effective.
- b. When the denture gets dirty or accumulates plaque, clean by using a denture brush and dentifrice.

STORAGE

Avoid high temperature and high humidity. Keep away from direct sunlight.

MOULDS

ENDURA	Maxillary/ Mandibular	Shape	Size
Anterior (ANTERIO)	Maxillary Mandibular	HS	3
		HT	4
		HO	5
		HC	6
		HSS	
Posterior (POSTERIO)	Maxillary Mandibular	S	28
		M	30
			32
Posterior (ZERO)	Maxillary Mandibular	M	28
			30
			32