- EN** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- RU** Инструкция по применению
- RO** Instrucțiuni de utilizare
- EL** Οδηγίες χρήσης
- DE** Gebrauchsanleitung
- NL** Gebruiksaanwijzing
- IT** Istruzioni per l'uso
- PL** Instrukcja obsługi
- SV** Bruksanvisning
- PT** Instruções de uso

04:2019-03
300071412

Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

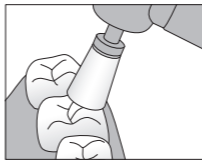
SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684


Cup PN 0181

For
Occlusal,
Cusp,
Incisal


Midi PN 0182

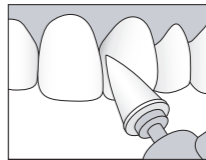For
Labial,
Lingual,
Buccal,
Occlusal

IC PN 0183

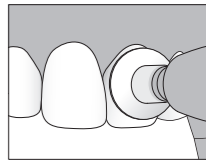For
Labial,
Lingual,
Buccal,
Interproximal

Mini-point PN 0186

For
Fissure,
Cervical

OneGloss Mandrel CA PN 0435

For
OneGloss
unmounted
polishers

Do not re-use
OneGloss unmounted
polishers (Cup, Midi, IC)

Maximum permissible speed
15,000 min⁻¹

EN READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

OneGloss is a one-step finisher and polisher that provides the ideal finish for composite restorations, simply by altering the contact pressure as you go from finishing to polishing.

INDICATIONS FOR USE

- Finishing and polishing of composite restoration
- Removal of excess resin or cement on tooth surface
- Removal of cements and polishing of enamel after removing and debonding orthodontic brackets
- Polishing of enamel after tooth scaling
- Removal of stain from tooth surface

PRECAUTIONS AND WARNINGS

1. OneGloss unmounted polishers (Cup, Midi, IC) are designed for single use only. Discard these polishers after use in accordance with the disposal protocol of your healthcare facility.
2. Do not use damaged or defective polishers or mandrel, i.e. deformation, dirt, corrosion, and exposure of shank due to repeated use.
3. Use protective mask and eyewear when operating handpieces.
4. Do not use this product for any purposes other than specifically outlined in these instructions for use.
5. The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

1. Seat the unmounted polisher onto the dedicated mandrel, OneGloss Mandrel, securely.
 - Do not use unnecessary strong force and improper angle when seating the unmounted polisher onto the mandrel in order to avoid damages to the polisher.
2. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
3. Apply heavy pressure for finishing and feather light pressure for polishing.

Shape	Application	Recommended pressure	
		Finishing	Polishing
Cup	Cusp, incisal, occlusal	1.0 N	0.3 N
Midi	Occlusal, labial, lingual, buccal		
IC	Labial, lingual, buccal, interproximal		
Mini-point	Fissure, cervical		

➢ Use at rotation speed of between 3,000 min⁻¹ to 10,000 min⁻¹ with intermittent water spray. Avoid exceeding maximum permissible speed, 15,000 min⁻¹.

- Use saliva ejector or suction to eliminate moisture and debris during the procedure.
4. After using OneGloss Mandrel, Mini-point, remove any contaminants immediately, disinfect and sterilize. Dispose the unmounted polishers (Cup, Midi, IC) after each use.

STERILIZATION AND STORAGE

OneGloss polishers and mandrel should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Avoid external pressure (physical load) and contamination. When using intraorally, sterilize before use. For re-use of OneGloss Mandrel and Mini-point, remove any contaminants using suitable cleaning liquid or disinfectant and perform autoclave.

Product name	Autoclave
OneGloss Polishers	134 °C/273 °F - 3'00"
OneGloss Mandrel	121 °C/250 °F - 30'00"

Follow manufacturer's instructions for use of cleaning liquid, disinfectant or sterilizer. Chloride disinfectant should be avoided.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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EC REP
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

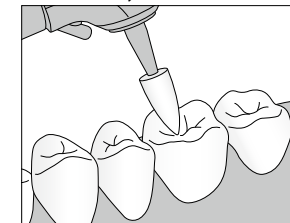
SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

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INDICATIONS FOR USE

- > Finishing and polishing of composite restoration
- > Removal of excess resin or cement on tooth surface
- > Removal of cements and polishing of enamel after removing and debonding orthodontic brackets
- > Polishing of enamel after tooth scaling
- > Removal of stain from tooth surface

PRECAUTIONS AND WARNINGS

1. Do not use damaged or defective instrument, i.e. deformation, dirt, corrosion, exposure of shank due to repeated use.
2. Use protective mask and eyewear when operating handpieces.
3. Do not use this product for any purposes other than specifically outlined in these instructions for use.
4. The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

1. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
2. Apply heavy pressure for finishing and feather light pressure for polishing.

Shape	Application	Recommended pressure	
		Finishing	Polishing
Mini-point	Fissure, cervical	1.0 N	0.3 N

- > Use at rotation speed of between 3,000 min⁻¹ to 10,000 min⁻¹ with intermittent water spray. Avoid exceeding maximum permissible speed, 15,000 min⁻¹.
 - > Use saliva ejector or suction to eliminate moisture and debris during the procedure.
3. After using OneGloss Mini-point, remove any contaminants immediately, disinfect and sterilize.

STERILIZATION AND STORAGE

OneGloss Mini-point should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Avoid external pressure (physical load) and contamination.

When using intraorally, sterilize before use. Remove any contaminants using suitable cleaning liquid or disinfectant and perform autoclave.

Product name	Autoclave
OneGloss Mini-point	134 °C/273 °F - 3'00" 121 °C/250 °F - 30'00"

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