Porcelain Veneer Kit

04.0:2020-09
72446

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

CE 0123

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Robot Points	300,000 min ⁻¹
T&F Hybrid Points	450,000 min ⁻¹
Dura-White Stones	30,000 min ⁻¹
Ceramisté	20,000 min ⁻¹

en MAXIMUM PERMISSIBLE ROTATION SPEED
de MAXIMAL ZULÄSSIGE DREHZAHL
fr VITESSE MAXIMALE DE ROTATION AUTORISÉE
nl MAXIMAAL TOELAATBARE ROTATIESNELHEID
es VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN
it MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
sv HÖGSTA TILLÅTNA ROTATIONSHASTIGHET
pt VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

en

READ CAREFULLY BEFORE USE

de

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nl

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PRODUCT DESCRIPTION

This kit is designed for making porcelain veneer placement. Kit has one Regular-grit and two Fine-grit FG Robot diamond points, three T&F Hybrid points, two Dura-White stones in FL2 and RE1 shapes, eight Ceramisté Standard (for prepolishing), Ultra (for polishing) and Ultra II (for superpolishing) in knife, wheel and cup shapes. Robot Points, T&F Hybrid Points and Dura-White Stones will finish porcelain smoothly. Ceramisté Polishers will polish and final polish porcelain veneers. Robot Points, T&F Hybrid Points, Dura-White Stones and Ceramisté Polishers are autoclavable for reuse.

PRECAUTIONS AND WARNINGS

●Never exceed the maximum permissible rotation speeds described in Table A. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use. ●Dispose in accordance with relevant local and national regulation.

STORAGE AND STERILIZATION

●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●Dura-White Stones: Do not perform plasma sterilization. ●For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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