

ZiLMaster


The Key to Polishing Modern Ceramics



ZiLMaster

ZiLMaster is a colour co-ordinated, three-step diamond impregnated polisher, specially designed with a uniform distribution of diamond particles in a resilient silicone binder to provide the smoothest surface finish and high gloss polish on modern all-ceramic restorations such as **Zirconia** and **Lithium Disilicate**.

ZiLMaster comes in a variety of shapes in 3 specific grits for both intra-oral and chairside application.

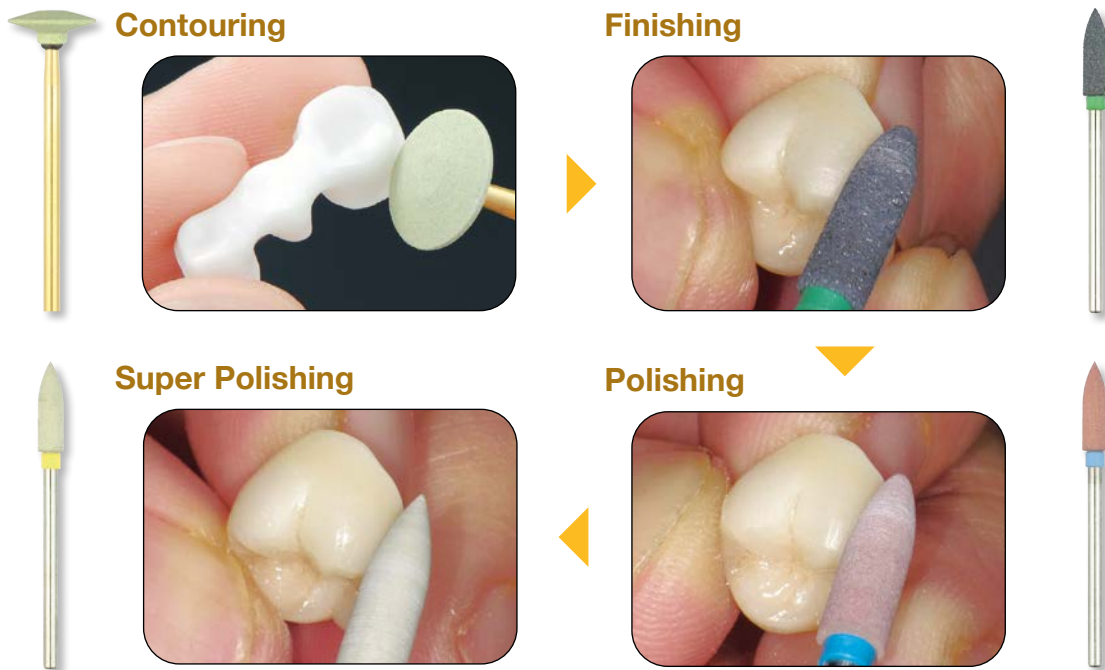


CONTOURING WITH DURA-GREEN DIA


Contouring of oxide-ceramics such as Zirconia involves shaping as well as the removal of larger surface scratches. Dura-Green DIA is a specially developed vitrified diamond impregnated abrasive that yields a smoother surface with virtually no chipping and less heat generation.



Surface image (x 400) after contouring with Dura-Green DIA and Diamond Point Fine

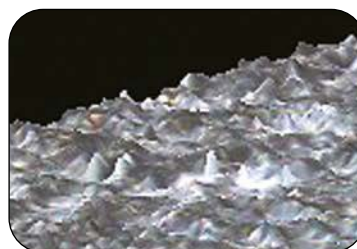


Contoured with Dura-Green DIA

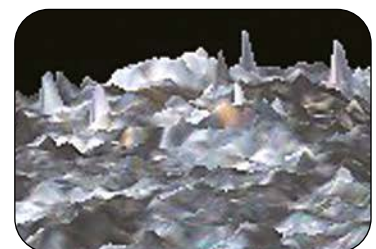

Contoured with Diamond Point Fine

PREDICTABLE STEPS TO A LASTING GLOSS

Material	Extra-oral / Intra-oral	Recommended procedure from contouring to polishing			
		Contouring	Finishing	Polishing	Super Polishing
Zirconia	Extra-oral Intra-oral	Dura-Green DIA	ZiLMaster Coarse	ZiLMaster Medium	ZiLMaster Fine
Lithium Disilicate (or other ceramics)	Extra-oral Intra-oral		*Only in delicate areas such as margins or edges of inlays/onlays/ laminare veneers.	ZiLMaster Medium	

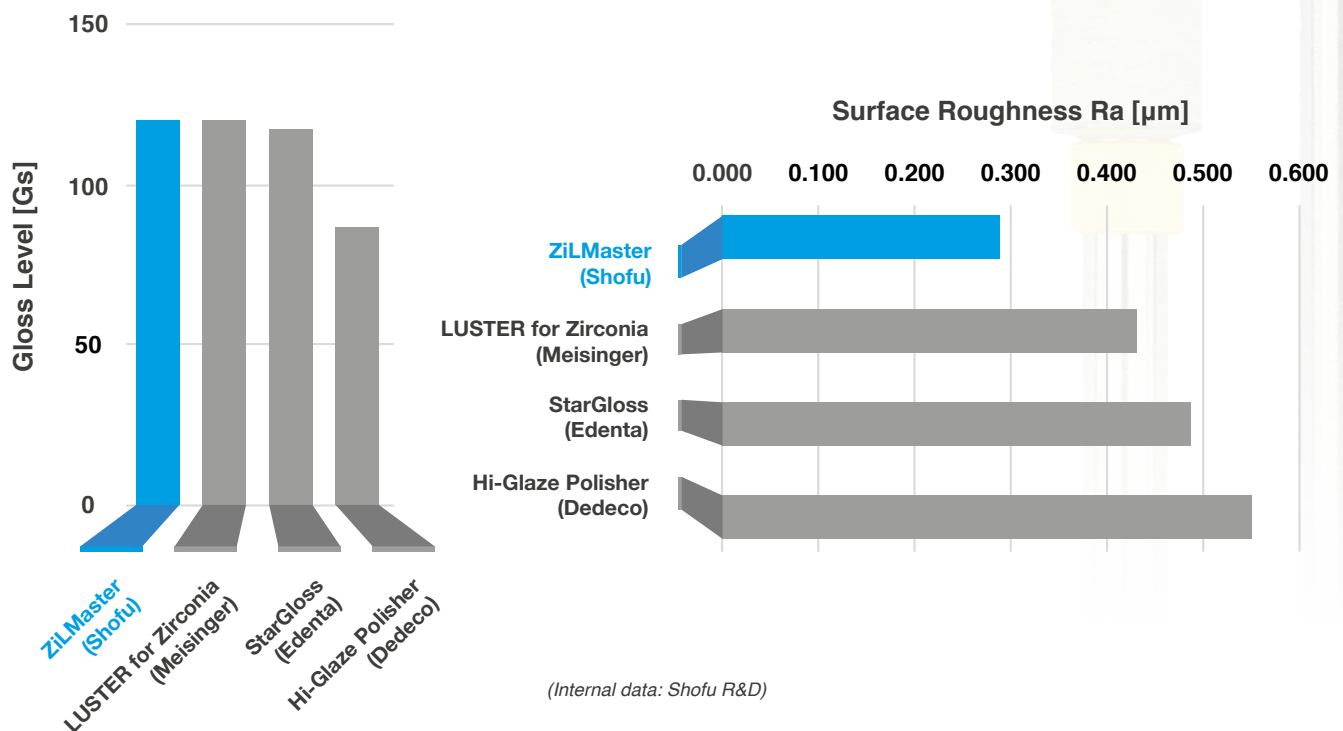

ZILMASTER FINISHING & POLISHING KIT HP

A comprehensive and resilient, diamond impregnated finishing and polishing system for lab / chairside use, containing an assortment of Dura-Green DIA and colour-coded **ZiLMaster** Polishers in 3 distinct grits that provide the highest natural lustre on all types of modern ceramics including Zirconia and Lithium Disilicate restorations.



	Shape	Bullet	Knife	Cup	Minipoint	KN7	WH6	Unmounted Wheel
	ISO NO.	040	090	065	030	125	120	170
ZiLMaster Coarse	HP 3 pcs	0640			0641	0642	0643	
	HP 1 pcs							0645
	CA 3pcs	0658	0662	0663	0661			
ZiLMaster Medium	HP 3 pcs	0646			0647	0648	0649	
	HP 1 pcs							0651
	CA 3 pcs	0665	0659	0668	0666			
ZiLMaster Fine	HP 3 pcs	0652			0653	0654	0655	
	HP 1 pcs							0657
	CA 3 pcs	0670	0672	0673	0671			

Recommended Speed Range (Mounted Polisher) : 10,000 – 15,000 rpm / Max. 20,000 rpm, Contact Pressure : 1.5 N
Recommended Speed Range (Unmounted Wheel) : 5,000 – 10,000 rpm / Max. 15,000 rpm, Contact Pressure : 1.5 N

ORDER INFORMATION



ZILMASTER FINISHING & POLISHING KIT HP [REF S0009]	
TYPE	SHAPE
Dura-Green DIA	TC4
	KN7
	WH6
ZiLMaster Course	Bullet
	KN7
	WH6
ZiLMaster Medium	Bullet
	KN7
	WH6
ZiLMaster Fine	Bullet
	KN7
	WH6
ZiLMaster Refills 1 or 3 pcs / box	HP and CA Shank

ZILMASTER ASSORTED	
GRIT / SHANK	REF
Coarse CA Bullet, Knife, Cup & Minipoint	0664
Medium CA Bullet, Knife, Cup & Minipoint	0669
Fine CA Bullet, Knife, Cup & Minipoint	0674
Coarse HP Bullet, KN7, WH6 & Minipoint	0644
Medium HP Bullet, KN7, WH6 & Minipoint	0650
Fine HP Bullet, KN7, WH6 & Minipoint	0656

• Unmounted Wheel is available individually with 1 pc of mandrel

DURA-GREEN DIA		
GRIT / SHANK		REF
HP	CY5	0156
	FL3	0158
	IC5	0162
	KN7	0154
	RD3	0159
	TC2	0161
	TC4	0155
	WH6	0157
CA	FL3	0163
	RD3	0164
	TC2	0165
HP Assorted	CY5	0153
	KN7	
	TC4	
CA Assorted	FL3	0166
	RD3	
	TC2	



SHOFU DENTAL ASIA-PACIFIC PTE. LTD.
 1 Science Park Road, #01-01 / 02 The Capricorn, Science Park II,
 Singapore 117528 Tel: (65) 6377 2722
 eMail: mailbx@shofu.com.sg www.shofu.com.sg



ZIL20250630