BEAUTIFIL Injectable X

BEAUTIFIL Injectable X SL

EN Instructions for use

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01:2019-07
72097

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore

EN

READ CAREFULLY BEFORE USE

Light-cured Dental Restorative Material

PRODUCT DESCRIPTION

This product is a light-cured radiopaque restorative material indicated for anterior and posterior restoration including occlusal surfaces. Available in 2 distinct viscosities: BEAUTIFIL Injectable X (Universal) and BEAUTIFIL Injectable XSL (Self-Leveling).

INDICATIONS FOR USE

- Anterior and posterior restorations
- Repair of restorations and prostheses

PRECAUTIONS AND WARNINGS

- Do not use this product on patients with known allergies to this material and/or methacrylate monomer.
- Operators with known allergies to this material and/or methacrylate monomer must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- Use medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers can penetrate through the gloves very quickly to reach the skin. Therefore, should this material contact even the gloves, immediately remove them and wash hands with plenty of water.
- In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Do not re-use needle tips.
- Follow the instructions for use of each material, instrument or equipment to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container.
(Example: YYMM-DD → Year-Month-Date of the expiration date)
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Preparation

When storing these materials in a refrigerator, allow them to reach room temperature before use. Attach the needle tip to the syringe as indicated in the figures below.

- Pull off the shield cap from the syringe while turning it slowly (Fig. A).
- Attach the needle tip by turning it clockwise until it is securely attached (Fig. B).
- Insert the needle tip into the small hole on the head of the shield cap immediately before use to protect the material from ambient light (Fig. C).
 - When replacing the needle tip, always use clean medical (dental) gloves.

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- Cleaning of tooth surface**
Thoroughly clean the tooth surface to remove plaque.
- Shade selection**
Select appropriate shade using a shade guide while the tooth is still moist.
- Cavity preparation**
Prepare the cavity following the standard method.
- Isolation**
Use a rubber dam etc to isolate the restoration site.
- Pulp protection**
In the case where the pulp is exposed or the affected area is close to the pulp, cap with calcium hydroxide etc.
 - Do not use eugenol containing materials since they might inhibit polymerization or adhesion of the material.
- Conditioning and application of adhesive**
Apply dental adhesive following the manufacturer's instructions for use.

A-7. Filling

Apply necessary amount of the material directly into the cavity and create the shape desired.

- When the cavity is deep, incremental filling and light curing of the material, in layers not exceeding 2 mm each, is recommended.
- Discard the used needle tip after use and replace the shield cap securely. Needle tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination. Do not re-use.
- Disinfect the syringe and shield cap with alcohol after each use to avoid cross-contamination. Then, close the cap securely. Use caution to avoid including alcohol into the paste.

A-8. Light-curing

Light-cure with a dental light-curing unit. In cases where the restored area is large, light-cure in segments.

Light-curing unit	Halogen	LED
Light-curing time	20 sec.	10 sec.
Wavelength	400-500 nm	440-490 nm
Light intensity	≥ 500 mW/cm ²	≥ 1000 mW/cm ²

- Use a dental light-curing unit with above specifications.
- Operators must use protective glasses or light shielding plate to avoid direct exposure to curing light. Operators must also protect patients' eyes from direct exposure to curing light.
- Disinfect the light-curing unit with alcohol after each use to avoid cross-contamination.

A-9. Finishing and polishing

After light-curing, contour, finish and polish the restoration following the standard method.

B. Repair of restorations and prostheses

- Clean the restoration**
Thoroughly clean the restoration to remove plaque.
- Shade selection**
Select appropriate shade using a shade guide.
- Roughing of the restoration**
Roughen the surface to be repaired with a dental abrasive (diamond burs etc). Then, rinse with water and dry.
- Isolation**
Use a rubber dam etc to isolate the restoration site.
- Pretreatment of the restoration**
Pretreat the material to be repaired (composite resin, porcelain, metal or natural tooth) to enhance mechanical and chemical retention. Follow the instructions of each pretreatment material for application and procedures of etching, priming and bonding.
- Filling, light-curing, finishing and polishing**
 - Follow the procedures from A-7 to A-9 above.

COMPOSITION

Bis-GMA, TEGDMA, Bis-MPEPP, S-PRG filler based on fluoroboroaluminosilicate glass, Polymerization initiator, Pigments and others

SHADES

BEAUTIFIL Injectable X: A1, A2, A3, A3.5, A4, B1, B2, C2, D2, A10, A20/
SA0.5, SA0.5O, Inc (Incisal), BW (Bleaching white)

BEAUTIFIL Injectable XSL: A1, A2, A3, A3.5, A4, B1, B2, C2, D2, A20, A30/
SA0.5, Inc (Incisal), BW (Bleaching white), CV (Cervical), MI (Milky)

STORAGE

Store in a dark place at room temperature (1-30 °C). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

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SA0.5, SA0.50, Inc (Incisal), BW (Bleaching white)