# Super-Snap Buff Disk

03:2019-03  
300030528

Manufacturer

SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

EC REP

SHOFU DENTAL GmbH  
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

CE 0123

## MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Super-Snap Buff Disks	10,000 min <sup>-1</sup>
-----------------------	--------------------------

## AUTOCLAVE : Metal mandrel (Table B)

Temperature	Time
134 °C/273 °F	3'00"
121 °C/250 °F	30'00"

DE MAXIMAL ZULÄSSIGE DREHZAHL  
FR VITESSE MAXIMALE DE ROTATION AUTORISÉE  
NL MAXIMAAL TOELAATBARE ROTATIESNELHEID  
ES VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN  
IT MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA  
SV HÖGSTA TILLÄTNA ROTATIONSHASTIGHET  
PT VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

DE AUTOKLAV : Metallmandrell  
FR AUTOCLAVE : Mandrin métallique  
NL AUTOCLAAF : Metalen mandrel  
ES AUTOCLAVE : Mandril metálico  
IT AUTOCLAVE : Mandrino metallico  
SV AUTOKLAV : Metallmandrell  
PT AUTOCLAVE : Mandril de metal

DE AUTOKLAV : Metallmandrell  
FR AUTOCLAVE : Mandrin métallique  
NL AUTOCLAAF : Metalen mandrel  
ES AUTOCLAVE : Mandril metálico  
IT AUTOCLAVE : Mandrino metallico  
SV AUTOKLAV : Metallmandrell  
PT AUTOCLAVE : Mandril de metal

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READ CAREFULLY BEFORE USE

EN

For composite, porcelain or gold restorations. Super-Snap Buff Disks have no metal centers or exposed mandrels that can discolor or gouge composite materials. It features a synthetic velvet polishing cloth outer surface which retains polishing pastes with minimum splatter or waste.

## PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Do not re-use Super-Snap Buff Disk.
- Read all instructions on the back side carefully before use.

## STERILIZATION

- Metal mandrel: Perform autoclave under the conditions described in Table B.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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#### (CAUTIONS)

●Never exceed the maximum permissible rotation speeds. ●Do not re-use Super-Snap Buff Disk. ●Mount a disk onto the metal mandrel provided securely. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective disk or mandrel i.e. deformation, dirt corrosion or surface flaw. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Use local dust extractor and protective equipment while using. ●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use.

#### (STERILIZATION: Metal mandrel)

●Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization. ●When using intraorally, sterilize before each use. ●For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize. ●Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. ●Do not perform dry heat sterilization. ●Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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