
- en** Instructions for use
- fr** Mode d'emploi
- es** Instrucciones de uso
- ru** Инструкция по применению
- ro** Instrucțiuni de utilizare
- el** Οδηγίες χρήσης
- de** Gebrauchsanleitung
- nl** Gebruiksaanwijzing
- it** Istruzioni per l'uso
- pl** Instrukcja obsługi
- sv** Bruksanvisning
- pt** Instruções de uso

COMBINATION CHART

Shade group	W	A					B				C				D		
Tooth shade	W3	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
VINTAGE LD PRESS MT-ingot	MT-W3	MT-A1	MT-A2	MT-A3	MT-A3.5	MT-A4	MT-B1	MT-B2	MT-B3	MT-B4	MT-C1	MT-C2	MT-C3	MT-C4	MT-D2	MT-D3	MT-D4
VINTAGE LD PRESS T-ingot	-	T-1	T-1	T-2	T-2	T-3	T-1	T-1	T-2	T-3	T-1	T-3	T-3	T-3	T-1	T-1	T-3
VINTAGE LD PRESS LO-ingot	-	LO-1	LO-2	LO-2	LO-2	LO-4	LO-1	LO-1	LO-2	LO-2	LO-1	LO-3	LO-3	LO-4	LO-1	LO-5	LO-5
VINTAGE LD PRESS MO-ingot	-	MO-1	MO-2	MO-2	MO-2	-	MO-1	MO-1	MO-2	MO-2	MO-1	MO-3	MO-3	-	MO-1	-	-
VINTAGE LD Opaque Dentin	OD-A1:1 OD-N:1	OD-A1	OD-A2	OD-A3	OD-A3.5	OD-A4	OD-B2:1 OD-N:1	OD-B2	OD-B2:1 OD-B4:1	OD-B4	OD-C2:1 OD-N:1	OD-C2	OD-C2:1 OD-C4:1	OD-C4	OD-D3:2 OD-N:1	OD-D3	OD-D3:1 OD-B4:1
VINTAGE LD Body	W3B	A1B	A2B	A3B	A3.5B	A4B	B1B	B2B	B3B	B4B	C1B	C2B	C3B	C4B	D2B	D3B	D4B
VINTAGE LD Enamel	OPAL 56:1 OPAL 57:1	OPAL 57	OPAL 58	OPAL 59	OPAL 59	OPAL 60	OPAL 57	OPAL 58	OPAL 59	OPAL 60	OPAL 58	OPAL 58	OPAL 59	OPAL 60	OPAL 57	OPAL 58	OPAL 59

FIRING SCHEDULE

Type of firing	Drying and Preheating furnace entrance (min)	Temperature rising speed (°C/min)	Firing schedule
Opaque Dentin	4 to 5	40 to 60	
Body			
Enamel			
Gum			
Self-glaze	4 to 5	40 to 60	
Correction	4 to 5	40 to 60	

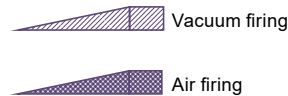PHYSICAL PROPERTIES

Type of firing	Coefficient of thermal expansion (25-500°C)	Glass transition point (°C)	
Opaque Dentin	2.Firing	9.3×10 ⁻⁶ K ⁻¹	512
	4.Firing		
Body	2.Firing	9.3×10 ⁻⁶ K ⁻¹	512
	4.Firing		
Enamel	2.Firing	9.3×10 ⁻⁶ K ⁻¹	512
	4.Firing		
Gum	2.Firing	9.4×10 ⁻⁶ K ⁻¹	506
	4.Firing		
Correction	2.Firing	8.5×10 ⁻⁶ K ⁻¹	490
	4.Firing		

Solubility test (ISO specification: below 100µg/cm²)

	Solubility amount(µg/cm ²)
Body	5
Enamel	2

SYSTEM AND SHADES

System	Shades
Opaque Dentin (11 shades)	OD-A1, OD-A2, OD-A3, OD-A3.5, OD-A4, OD-B2, OD-B4, OD-C2, OD-C4, OD-D3, OD-N
Body (17 shades)	W3B, A1B, A2B, A3B, A3.5B, A4B, B1B, B2B, B3B, B4B, C1B, C2B, C3B, C4B, D2B, D3B, D4B
Enamel	Enamel (5 shades) OPAL 56, OPAL 57, OPAL 58, OPAL 59, OPAL 60
	Enamel Effect (10 shades) OPAL T, OPAL SL, T, BT, PT, GT, YT, OT, AM-Y, T-GLASS
Gum (5 shades)	Gum-1, Gum-2, Gum-3, Gum-4, Gum-5
Correction (2 shades)	ADD-ON B, ADD-ON T

en Lithium Disilicate Bond Porcelain

READ CAREFULLY BEFORE USE

VINTAGE LD is a dental ceramic to be layered over Lithium disilicate press glass ceramic, VINTAGE LD PRESS. Body shades are available in classical shades as well as a variety of effect, translucent and opalescent incisal shades. VINTAGE LD is a low fusing porcelain and can be fired at 800 °C or lower.

INDICATIONS

Build up porcelain for fabrication of all ceramic restorations including lithium disilicate

PRECAUTIONS

- If any inflammation or other allergic reactions occur on operator, immediately discontinue use and seek medical advice.
- Avoid contact with eyes. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Wear protective glasses etc. while grinding and polishing this product to avoid damages to eyes.
- Use local dust extractor, dust protective mask, etc. while grinding this product to avoid harmful influence of the dust on the human body.
- This product should not be used for the cases where residual abutment tooth is too small, or for the patients with bruxism or occlusal abnormality etc.
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Refer to the separate instruction manual or step card.

NOTE ON USE

- Final form and size should be restored with body porcelain to build up opalescent enamel or translucent porcelain accurately.
- Drying time should be altered depending on the size of the restoration to be fired. (Restorations might explode inside the furnace if moisture remains inside.)
- Enamel porcelain should be built up oversized in consideration of firing shrinkage.
- When building up additional porcelain onto the dried area, wet the area with water moistened brush in advance.
- Optimum firing schedule is different depending on the type and model of the furnace. Try to determine the optimal conditions at which every restoration is fired completely.
- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- Do not use this product in conjunction with PFMs, metal frames or ceramic frames (alumina).
- Do not use this product for any purpose other than specifically outlined in INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container. (Example ☞ YYYY-MM-DD → Year-Month-Date of the expiration date)

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high temperature and high humidity. Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

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