



BLUE INLAY WAX

EN Instructions for use

02:2015-09
18922



Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EN Dental casting wax

INDICATIONS

Direct intraoral wax up for inlays, crowns and bridges

COMPOSITION

Paraffin wax, Carnauba Wax, Dammer, Beeswax and others

SPECIFICATION

Item	Internal specification
Melting point	59 °C/138 °F
Residue on ignition ^{*1}	0.1% or less

*1 : Tested in accordance with JIS T 6503:2005

PRECAUTIONS

1. This product is intended for use by dental professionals only.
2. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.

DIRECTIONS FOR USE

1. Heat this material over a flame to adequately soften it in a conventional manner.
2. Apply the softened wax over the occlusal surface of the prepared tooth and have the patient to bite on it to register correct occlusion.
 - Use caution to avoid accidental ingestion by patient while manipulating this material in patient's mouth.
3. After the wax is solidified, trim the excess material using an instrument to obtain adequate contour.

NOTE ON USE

1. Use caution to avoid catching fire while using this material since it is flammable.
2. Do not overheat or leave this product while heating.

STORAGE

Store at room temperature (1-30 °C /34-86 °F).
Keep away from direct sunlight and any source of ignition.