

BeutiSealant

CLICK / SCAN



FOR MORE INFO

BIOACTIVE PIT & FISSURE SEALANT

Fast & predictable fluoride releasing Pit & Fissure Sealant for enhanced caries prevention

- **Self-etching Primer** – non-rinse, gentle on tooth & strong chemical bond
- **Bioactive Paste** – precise, bubble free uniform delivery to seal deep grooves & fissures efficiently

The patented Bioactive S-PRG filler technology provides a fluoride rich environment with an anti-plaque effect to protect against caries overtime.

BeutiSealant, completely eliminates the need for phosphoric acid etching saving chair time with faster, simpler application while ensuring predictable outcomes.



Available in:

- **BeutiSealant Full Set**
BeutiSealant Paste 1.2g, BeutiSealant Primer 3ml, Needle Tips x 15, Microbrush Fine (pink) x 50, V-Dish x 25
- **Refills - BeutiSealant Paste 1.2g**

CLINICAL APPLICATIONS

- Deep pits and fissures in newly erupted teeth (less than 4 years)
- High caries risk patients through the caries active period (ages 6 to 15years)
 - 3-4 years - Primary Molars
 - 6-7 years - 1st Permanent Molar
 - 11-13 years - 2nd Permanent Molar
- During Orthodontic treatment
- Medically compromised patients
- Non cavitated / initial occlusal caries in adults

FEATURES

- ✓ **Shorter chair time** – Non-rinse Priming just *apply, leave and dry*
- ✓ **Convenient** – 360° finger grip for multidirectional application
- ✓ **Ideal consistency** – Highly flowable to reach deep /narrow fissures
- ✓ **Lasting seal** – Excellent mechanical properties
- ✓ **Radiopacity similar to dentin** – Evaluation and monitoring
- ✓ **Precise bubble-free delivery** – Extra fine needle tip (27 gauge)
- ✓ **Minimize wastage** – Unique syringe design with non-drip pull-back mechanism

DIRECTIONS FOR USE

- 4 simple steps
 1. Apply Primer and leave for 5 sec
 2. Gently air dry for 5 sec
 3. Apply the Sealant
 4. Light cure for 10 sec (with LED curing light)

RELATED PRODUCTS

- **PRG-Barrier Coat**
– Light cure Bioactive Protective Shield

