

# HY-BOND GLASIONOMER **CX-Smart**

CLICK / SCAN



FOR MORE INFO

## SELF-ADHESIVE LUTING CEMENT

Truly biocompatible, quick-set, luting glass ionomer that guarantees...

- **Reliable Cementation** with long-term stability
- **Perfect Fit** with uniform film thickness & moisture tolerance

Comprises of **SHOFU's** proprietary additive - HY-agent, "a tannic acid-strontium fluoride-zinc fluoride complex" known to:

- Promote remineralization
- Increase acid resistance
- Reduce solubility
- Eliminates post-operative sensitivity



### Available in:

- **Hy-Bond Glaslonomer CX-Smart 1-1 Set**  
Powder 45gm / Liquid 30gm (25ml)
- **Hy-Bond Glaslonomer CX-Smart Mini Set**  
Powder 15gm / Liquid 10gm (8ml)

## CLINICAL APPLICATIONS

Ideal for Cementation of

- Metal inlays, onlays, crowns and bridges
- Porcelain Fused to Metal (PFM) restorations
- Zirconia crowns & bridges
- Metal orthodontic brackets
- Metal post-core

## FEATURES

- ✓ **Reduce chair time & Increase patient comfort**
  - Self-adhesive
  - Fast-set in the mouth
  - Effortless removal of excess cement
  - Eliminates post-operative sensitivity
- ✓ **Provides long-term stability with**
  - High acid resistance
  - Low oral solubility
  - Excellent marginal seal
- ✓ **Cost effective, easy to use & reliable**

## DIRECTIONS FOR USE

- Powder / Liquid ratio – 1 scoop : 2 drops
- Working time – 30 sec
- Working time – 2 min (*from start of mixing*)
- Nett setting time – 2 min 30 sec (*from end of mixing*)
- Removal of Excess – 1 min 30 sec (*after seating restoration*)

## RELATED PRODUCTS

- Chairside adjustment
  - Porcelain Adjustment kit HP
- Intra-Oral adjustment
  - Enamel Adjustment kit CA



Official Partner

