



Livera α

EN Instructions for use

01:2016-07
W000007



Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EN Acrylic teeth

Livera α are two-layer acrylic resin teeth with natural anatomical morphology (harmony form).

SHADES

16 colors: A1, A2, A3, A3.5, A4, B1, B2, B3, B4,
C1, C2, C3, C4, D2, D3, D4

COMPOSITION

Anterior

Enamel: PMMA, others

Base: PMMA, others

Posterior

Enamel: PMMA, others

Base: PMMA, others

INDICATIONS

Used as denture teeth to produce partial or full dentures

PRECAUTIONS

1. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
2. Use a dust extractor, dust protective mask etc. while cutting or polishing this product, to avoid harmful effects of dust on the human body.
3. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
4. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Produce full or partial dentures following the conventional method.

- Do not expose to fire when fabricating wax denture.
- Give mechanical retention when using thermoplastic denture base resins. Livera α does not bond to thermoplastic denture base resins.
- Give the following general instructions to patient:
Keep the denture clean. Gently remove food debris with a soft denture brush after eating. Using denture cleanser is more effective.

STORAGE

Avoid high temperature and high humidity. Keep away from direct sunlight.

MOULDS

Livera α	Maxillary/ Mandibular	Shape	Size
Anterior	Maxillary Mandibular	HT HSS HC	3 4 5 6
Posterior	Maxillary Mandibular	M	28 30 32 34