

CLICK / SCAN



FOR MORE INFO

PRG Barrier Coat

BIOACTIVE LIGHT-CURE PROTECTIVE SHIELD

Smart solution to revolutionize the preventive protocol with two key benefits

- **Seal against Dentin Hypersensitivity** – immediate and lasting sensitivity relief
- **Protect against Dental Caries** – effective prevention and management of caries

A single application of PRG Barrier Coat provides immediate and lasting sensitivity relief. The durable coat protects teeth from external irritants and exhibits resistance to daily toothbrushing.

The patented bioactive S-PRG filler technology provides a unique combination of 6 beneficial ions to provide a fluoride rich environment with an anti-plaque effect to protect against caries overtime.



Available in:

- **PRG Barrier Coat Mini-Kit**
Base 0.02mL x 8pcs, Active 0.9ml
Disposable brush x 8, holder x 1

CLINICAL APPLICATIONS

- **Dentin hypersensitivity due to**
 - Tooth wear – Attrition, Abrasion, Erosion, Abfraction
 - Gingival recession
- **Prevent and manage dental caries**
 - Early caries lesions (white spots)
 - Exposed root surfaces
 - Around orthodontic brackets
 - Interproximal area in crowded teeth
 - Teeth with clasps of removal partial dentures
 - Newly erupted permanent molars
 - Special needs and medically compromised patients

FEATURES

- ✓ **Immediate and lasting dentin hypersensitivity relief**
 - Effective up to 6 months with a single application
 - Increase patient comfort & quality of life
- ✓ **Simplified clinical procedure**
 - Fast and easy application
 - Invisible, only 15µ thin for better aesthetics
 - Self- adhesive, HEMA-free, Acetone-free and Alcohol-free
- ✓ **Prolonged protection**
 - Durable and resistant against daily tooth brushing
- ✓ **Bioactive to protect against caries**
 - Minimize plaque formation
 - Reduce cariogenic bacteria
 - Promotes remineralization with fluoride release and recharge

DIRECTIONS FOR USE

- Add 1 drop of Activator to the Base & mix
- Paint on clean tooth surface in a single direction & leave for 3 sec
- Light cure for 10 sec (with LED curing light)

RELATED PRODUCTS

- **BeautiSealant**
– Bioactive Pit & Fissure Sealant

