



# RED INLAY WAX

## EN Instructions for use

02:2015-09  
18914



Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

## EN Dental casting wax

### INDICATIONS

Wax up for inlays, crowns and bridges with indirect technique

### COMPOSITION

Paraffin wax, Carnauba Wax, Dammer, Beeswax and others

### PRECAUTIONS

1. This product is intended for use by dental professionals only.
2. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.

### SPECIFICATION

Item	Internal specification
Melting point	58 °C/136 °F
Residue on ignition <sup>*1</sup>	0.1% or less

\*1 : Tested in accordance with JIS T 6503:2005

### DIRECTIONS FOR USE

1. Apply separating agent (SHOFU Sep etc) to plaster model and artificial teeth. Soften this material in conventional manner and build up the softened material on the model using a wax spatula.
2. After the wax is solidified, trim the excess material using an instrument to obtain adequate contour.

### NOTE ON USE

1. Use caution to avoid catching fire while using this material since it is flammable.
2. Do not overheat or leave this product while heating.

### STORAGE

Store at room temperature (1-30 °C /34-86 °F).  
Keep away from direct sunlight and any source of ignition