

# CERAMAGE

CLICK / SCAN



FOR MORE INFO

## CERAMAGE FINISHING & POLISHING KIT

Designed to achieve a highly polished, glazed-like surface on indirect composites & hybrid ceramics with ...

- Precise adjustments and predictable steps to fast and efficient polishing
- Special polishing waxes designed to achieve a blemish-free natural lustre



### The kit contains:

- Robot Carbide Fissure Bur HP x1
- Silicone Points HR2  
(1 each x 11, 13S & 2 pc x 28)
- SHOFU Pivot Brush x 1
- SHOFU Pivot Brush SC x 1
- SHOFU Mandrels x 2
- Dura-Polish 20g x 1 jar
- Dura-Polish DIA 5g x 1 jar
- SHOFU Felt Wheels  
(No.1602 - 16 mm diameter, 2 mm thickness) x 2
- SHOFU Felt Wheels  
(No.2405 - 24 mm diameter, 5 mm thickness) x 1

### APPLICATIONS

- Predictable polishing of indirect composites & hybrid ceramics
- Dura Polish – polishing of acrylic restorations
- Dura Polish DIA – High gloss polishing of porcelain restorations

### FEATURES

- ✓ Assortment of carefully selected polishers for sequential polishing to attain a lasting high gloss polish
- ✓ Fast, predictable and consistent results

### RECOMMENDED POLISHING SEQUENCE

1. Adjust fine details on occlusal surface with Robot Carbide Fissure Bur
2. Eliminate scratches and pre-polish with SHOFU Silicone Point HR2 and SHOFU Pivot Brush SC
3. Polish with SHOFU Pivot Brush using Dura-Polish & Dura Polish DIA respectively
4. Super-polish to a natural, high lustre with SHOFU Felt Wheel

### MAXIMUM RECOMMENDED SPEED

- Robot Carbide Fissure Bur: 50,000 rpm
- SHOFU Silicon Point: 20,000 rpm
- SHOFU Pivot Brush: 10,000 rpm
- SHOFU Pivot Brush SC: 10,000 rpm
- SHOFU Felt Wheel: 10,000 rpm

### RELATED PRODUCTS

- Ceramage Standard Set
- Ceramage Gum Color Set
- SHOFU Block HC & Disk HC

