

OneGloss

CLICK / SCAN



FOR MORE INFO

2-IN-1 IMPREGNATED SMART POLISHER THAT DOES IT ALL

Designed for clinicians who prefer the convenience of an all-in-one solution to shape, finish and polish direct restorations with one instrument.

Finish and Polish all tooth coloured direct restorative materials by simply altering the contact pressure. More pressure with intermittent water for finishing and feather light pressure for polishing.

- **3 convenient snap-on shapes designed for**
 - o Cup – cusp, incisal, occlusal surfaces
 - o Midi – occlusal, labial, lingual, buccal surfaces
 - o IC – labial, lingual, buccal, interproximal area
- **1 mounted point (re-useable)**
 - o Minipoint – occlusal fissures, cervical area



Available in:

- OneGloss Set (REF 0180)
20 x unmounted polishers in each shape,
3 x Mandrels CA
- Refills in a box of 50's
- Mandrels CA in box of 6's
- Mounted Mini Point in box of 12's

	Shape	Cup	Midpoint	IC	Minipoint	Mandrel
	ISO NO.	060	060	100	030	
OneGloss		0181	0182	0183	0186	0435

CLINICAL APPLICATIONS

- Finishing and polishing of all tooth coloured restorations
- Creating surface texture similar to natural teeth
- Removing surface stain without damaging the enamel
- Polishing enamel surface after scaling
- Removing excess cement
 - without marring enamel or damaging delicate inlays
 - after Orthodontic debanding

FEATURES

- ✓ **Disposable 2-in-1 Snap-on system** for fast, safe and predictable finishing & polishing with single instrument
- ✓ **Finishing** – use more pressure (approx 1.0 Newton) with intermittent water
- ✓ **Polishing** – use feather light strokes (pressure of approx 0.3 Newton)
- ✓ **Silicone points impregnated** with a high concentration of **aluminium oxide**
- ✓ **Durable** enough to withstand polishing an entire arch
- ✓ **Convenient**, easy to use and saves time
- ✓ **Recommended speed** (3.000 rpm – 10.000 rpm) with intermittent water spray. Avoid exceeding maximum permissible speed (15,000 rpm)

RELATED PRODUCTS

- SuperBuff Set
- Super-Snap X-TREME Disk
- DirectDia Diamond Polishing Paste
- Beautifil II / Beautifil II LS / Beautifil Injectable X



Official Partner

