

# PORCELAIN ADJUSTMENT KIT HP


CLICK / SCAN



FOR MORE INFO

## SPECIALLY DESIGNED FOR CHAIRSIDE FINE ADJUSTMENT AND POLISHING OF PORCELAIN RESTORATIONS BEFORE CEMENTATION

- Predictable steps to polishing
- Enables quick and efficient chairside adjustment to ensure patient comfort, while eliminating the need to send the restoration back to the lab
- Variety of shapes to adapt to all surfaces of the restorations

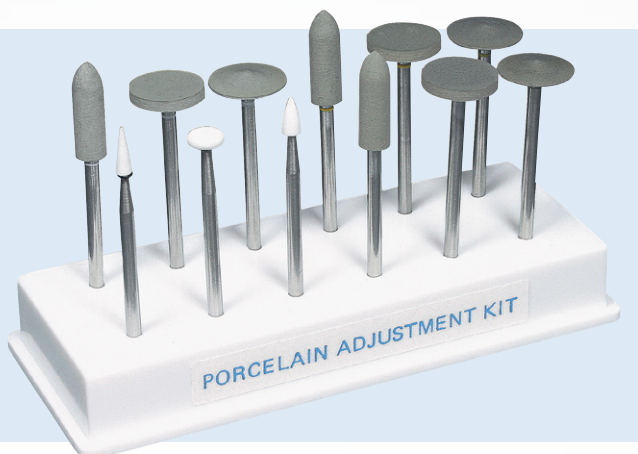

### The kit contains:

- Dura-White Stones HP – 1 each CN1, FL3 & RE1
- Ceramisté Points HP – 1 each of KN7, PC2 & WH6 in Standard, Ultra & Ultra II grits

## CLINICAL APPLICATIONS

- Minor adjustments, pre-polishing and polishing of porcelain fused to metal, porcelain fused to zirconia and feldspathic porcelain restorations – crowns, bridges, veneers, onlays

## FEATURES

- ✓ **Time-efficient**
  - Eliminates the need to send back to the lab for re-glazing
  - Saves chair time and avoids the need for recall visit
- ✓ **Predictable & Convenient**
  - Variety of shapes with easily identifiable, colour-coded shanks for sequential polishing
- ✓ **Cost-efficient & Safe**
  - Resilient and durable stones & polishers
  - Autoclavable & cost effective – prevents cross contamination and enables multiple use with total peace of mind

## RECOMMENDED POLISHING SEQUENCE

- Adjusting and Finishing – Dura-White Stones
- Polishing – Ceramisté Standard, Ultra (yellow band) & Ultra II (white band)

## MAXIMUM RECOMMENDED SPEED

- **Dura-White Stones:** 5,000 – 20,000 rpm
- **Ceramisté Points:** 10,000 – 12,000 rpm

## RELATED PRODUCTS

- Diamond Dresser
- DirectDia Polishing Paste
- Super-Snap Buff



Official Partner

