



ENHANCED POLISHING SYSTEM WITH 3D COATING TECHNOLOGY

Designed for clinicians who prefer a disposable disk system with a proven track record that enables hassle-free, safe and efficient finishing & polishing of all direct composite resins.

- **4 colour coded disks in 2 sizes (12mm and 8mm diameter)**
 - o Coarse Black: Contouring
 - o Medium Violet: Finishing
 - o Fine Green: Polishing
 - o Superfine Red: Super polishing
- Black / Violet disks are impregnated with silicon carbide. Green / Red disks are impregnated with aluminium oxide.

Super-Snap is a simple, fast and effective snap-on finishing and polishing disk system considered a “**must have**” in the restorative workflow. The double sided Super-Snap X-TREME Green and Red disks are durable and allows more pressure with a better tactile feel. The advanced 3D coating technology helps prevent clogging and secondary scratches to achieve a natural gloss with an exceptional surface lustre.



Available in:

- Super-Snap X-TREME Technique Mini Kit (REF N0509)
48 x standard disks, 1 x Composite Fine Midi-Point CA, 1 x Mandrel CA
- Refills in a box of 50's

Shape Category	Contouring		Finishing				Polishing		Super-Polishing	
Super-Snap Shape										
	Black (Coarse)		Violet (Medium)				Green (Fine)		Red (Super Fine)	
Refills (50's a box)	L506	L507	L508	L509	L528	L522	L501	L519	L502	L521

CLINICAL APPLICATIONS

- Finishing and polishing of all tooth coloured direct restorative materials specially in the anterior region

FEATURES

- ✓ **Snap-on, disposable disk system** for fast, safe and predictable finishing & polishing
 - **Ultra-thin flexible yet resilient** to easily reach interproximal areas with excellent visibility
 - **Soft, safe, elastic silicone snap-on** for easy placement on mandrels
 - **No metal centre** to completely eliminate the risk of gouging restorations
- ✓ **Dry polishing process** at low speed (10.000 rpm – 12.000 rpm) with light, intermittent strokes in a single direction for maximum control
- ✓ **Disposable disks** for enhanced infection control
- ✓ **Enhanced Super-Snap X-TREME** Green and Red disks provide
 - **Extra thickness** for durability and better tactile feel
 - **More elasticity, reliance and control** for an enhanced polishing experience
 - **Semi-spherical 3D surface structure** prevents clogging, reduces heat and secondary scratches
 - **A natural gloss** with an exceptionally smooth surface

RELATED PRODUCTS

- Super-Snap Buff Disk
- DirectDia Diamond Polishing Paste
- OneGloss 2-in-1 Smart Polisher
- Beautifil II / Beautifil II LS / Beautifil Injectable X