

RESIN GLAZE

CLICK / SCAN



FOR MORE INFO

PROTECTIVE, HIGH-GLOSS, LIGHT-CURE COAT

Specially developed light-cure temporary surface coat that seals margins and protects the surface with a natural gloss

- Naturally aesthetic glaze
- Simplifies the final finishing process
- Suitable to protect difficult to access areas from staining / discoloration
 - Pits & fissures
 - Interproximal surfaces

Optional Resin Glaze Primer is available for enhanced adhesion to various substrates, prior to application of Resin Glaze Liquid.



Available in:

- Resin Glaze Liquid, 6mL (REF Y0004)
- Resin Glaze Primer, 6mL (REF Y0003) - Optional

CLINICAL APPLICATIONS

- Indirect Composite
 - *Ceramage*
- CAD/CAM Hybrid Ceramics
 - *Shofu Block / Disk HC*
- CAD/CAM PMMA materials
- Acrylic resin denture prosthesis
- Artificial teeth
 - *Livera Alpha & Endura*
- Provisional resin restorations

FEATURES

- ✓ **Remarkable Colour Stability**
 - Minimizes discoloration after optimal light curing
- ✓ **High level of surface cure**
 - Minimal residual monomer (sticky layer)
- ✓ **Excellent Surface Gloss**
 - Aesthetic surface luster with just one application
- ✓ **Greater durability and wear resistance**
 - Higher degree of monomer conversion ensures uniform and deep cure

DIRECTION FOR USE

1. Finish and smoothen surface with SoftCut PB
2. Apply Resin Glaze Primer (Optional) & allow to self-dry for 10 sec
(To enhance bond strength)
3. Apply single layer of Resin Glaze Liquid
4. Light cure with MiCD Alpha-Cure LED 180 sec
(Applicable wavelength: 400 nm-410 nm)

RELATED PRODUCTS

- MiCD Alpha-Cure LED
- CeraResin Bond Set
- Lite Art Full Set

