

# BeautiBond Xtreme

CLICK / SCAN



FOR MORE INFO

## THE POWER OF ONE

**An all-in-one, truly universal adhesive for bonding direct & indirect restorations without the need of additional primers.**

- Strong, fast & reliable adhesion to dentin and enamel
- HEMA-Free formulation - easy removal of solvents, eliminates risk of allergies and bond hydrolysis, assuring a stable bond
- Flexibility for Self-Etch, Selective-Etch & Total-Etch
- Quadra-Functional Monomer Technology ensure predictable adhesion to Glass Ceramics, Feldspathic Porcelain & Lithium Silicate restorations
- Acid Resistance Silane (ARS) coupling agent developed with a protective module that prevents monomer degradation and guarantees predictable adhesion to ceramic restorations.



**Available in:**  
BeautiBond Xtreme Set (REF Y2448)  
(5ml Bottle, Micro brush 50's, V-dish 25's)

## CLINICAL APPLICATIONS

- Direct Composite Restorations – Class I – V, Non-carious defects (Erosion /Abrasion / Abfraction), Diastema closure, Direct Veneers
- Intra-oral repair of direct/ indirect restorations
- Cavity seal & Immediate Dentin Sealing (IDS)
- Immediate Hypersensitivity Relief
- Cementation of indirect restorations – Veneers / Crowns / Bridges / Inlays & Onlays

## FEATURES

- ✓ **Compatible with all DUAL-CURE/Light Cure Resin Cements**
  - No additional activator required
- ✓ **Safe & Reliable Adhesion**
  - Predictable adhesion with uniform bonding layer of 5 microns
- ✓ **Moisture Tolerant**
  - Bonds well to wet and dry condition
- ✓ **Streamline Inventory and save more**
  - Single bottle system with controlled dispensing without separate primers
- ✓ **Virtually no post-operative sensitivity**

## DIRECTION FOR USE

- 1. Apply the adhesive and leave**  
Rub for 20sec (in case of phosphoric acid etching)
- 2. Air Dry**  
Gentle Air (3 sec) followed by Strong Air
- 3. Light Cure**  
LED: 5 Sec / Halogen: 10 Sec

## RELATED PRODUCTS

- Beautifil Injectable X / XSL
- Beautifil II / II LS

