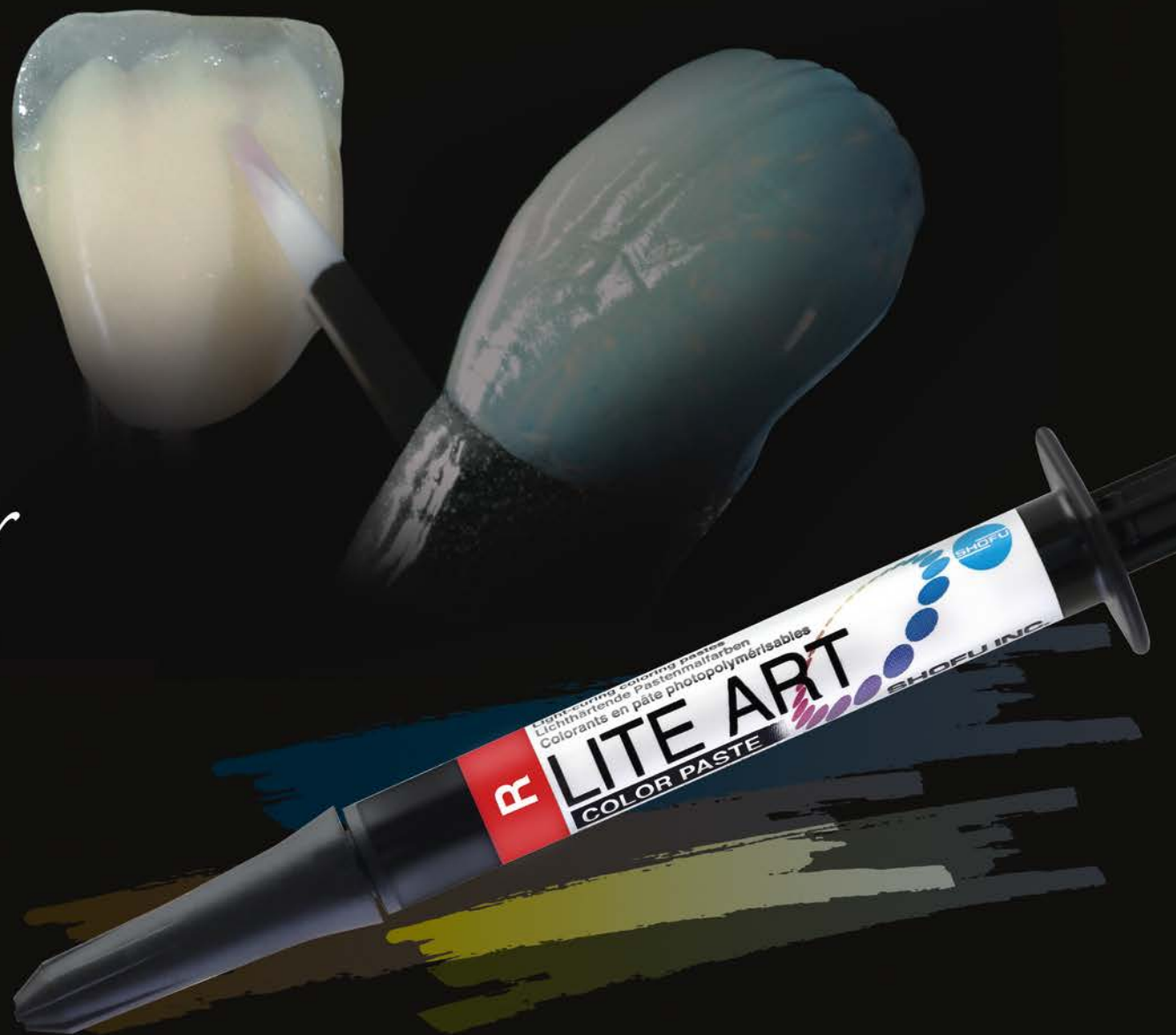
Stain Like a Master

Technique Guide

Copyright © From 2024 SHOFU DENTAL ASIA-PACIFIC PTE. LTD.
All rights reserved.

Credit by Ms. Nutcharee Ladcharoen

Recipe for VINTAGE Art Color using LITE ART CMY Color

Shades of VINTAGE Art can be created by mixing Cyan, Magenta, Yellow, White and Black of LITE ART.

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*The Yellow color of LITE ART differs somewhat from the Yellow color of VINTAGE Art.

●The colors in gray-shaded region in the mixing ratio are available in LITE ART system.

●The printed colors might be different from that of actual colors in this system.

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Shade Adjustment

A and B shades of color paste are done to match Classical shades.

They are used for the adjustment of chroma of tooth color restorations.

●After applying shade stain, the surface should be covered by translucent and so on.

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
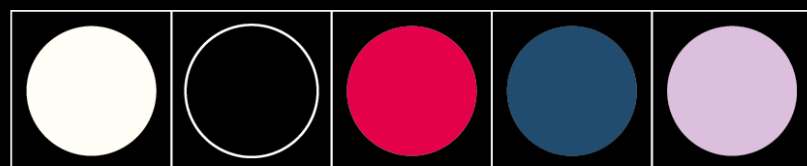
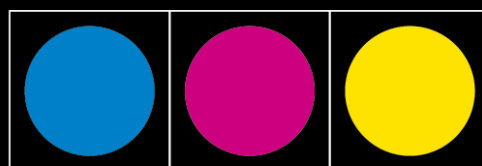
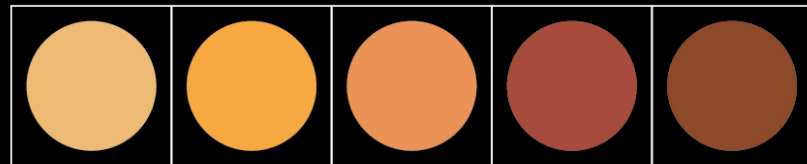
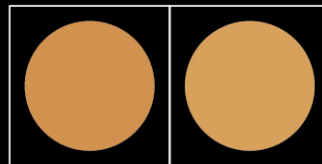
A Shade

B Shade

Shades of LITE ART (total 15 colors) are selected based on the popular shades in clinical use from SHOFU porcelain stain "VINTAGE Art" and are also originally created with distinctive features of organic pigments.

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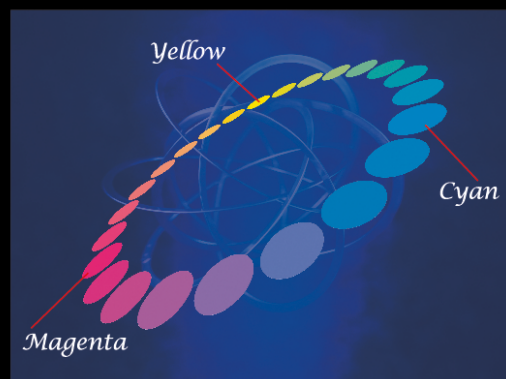What's CMY Color?

Red, Yellow and Blue are generally considered to be the three primary colors as they were also stated in classical study of chromatics. However, if they were mixed with each other, there were differences between the mixing ratio and the image of color to be created. And the range of coloration was also limited. Currently, in various fields such as color printing, a subtractive color mixing theory of three primary colors (Cyan, Magenta and Yellow) proposed by Ducos du Hauron in 1868, is used and, as a result, matching of color mixing ratio and image of color to be created became possible, and the range of individual coloration was also expanded.

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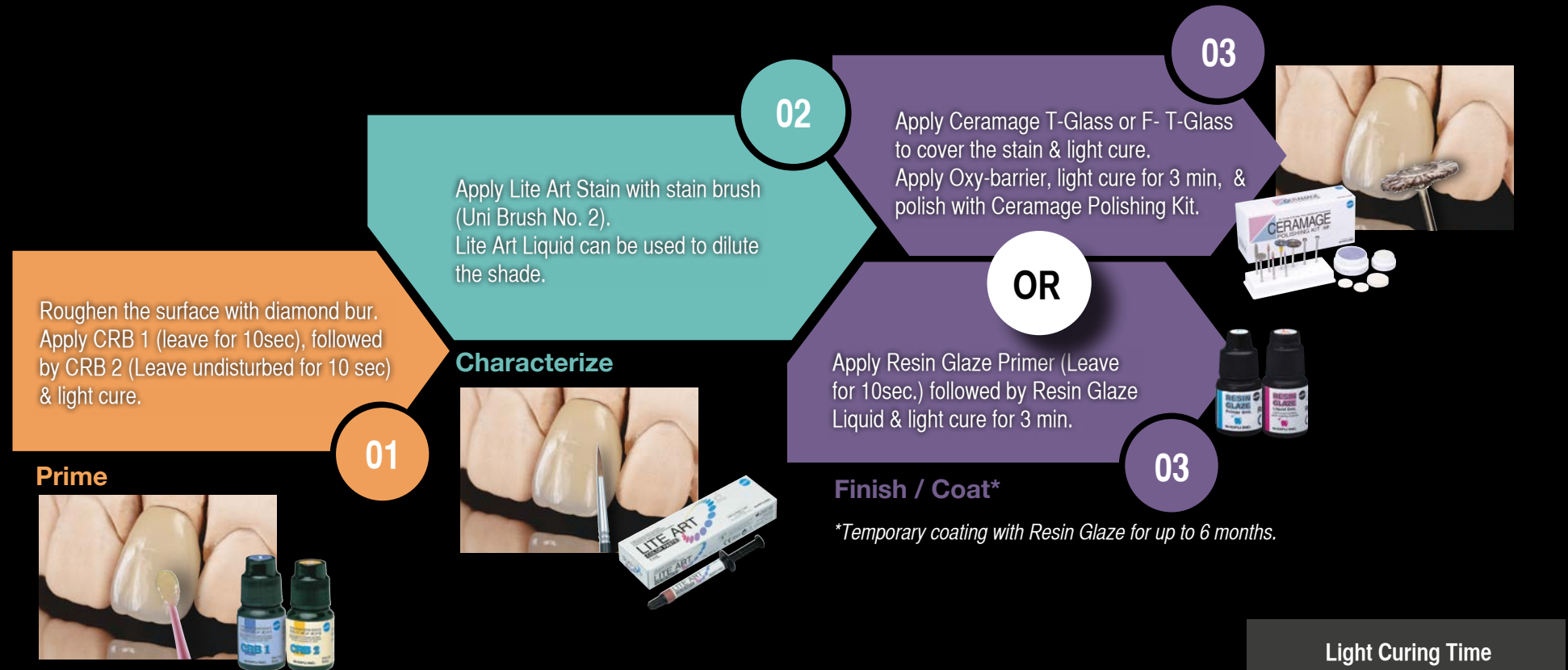
Build up with SHOFU Ceramage & light cure each layer for 1 min.

02

Apply Lite Art Stain with stain brush (Uni Brush No.2). Lite Art Liquid can be used to dilute the shade. Light cure- 1min.#

Characterize

Light Curing Time	
CRB 2	1 min
F-T-Glass	1 min
Lite Art	1 min
Resin Glaze Liquid	3 min
Oxy-barrier	3 min

03

Apply Ceramage T-Glass or F- T-Glass to cover the stain & light cure for 1 min.

Finalize

04

Apply Oxy-barrier and light cure for 3min. Finish and polish with Ceramage Polishing Kit.

Finish & Polish

OR

Apply Resin Glaze Primer (Leave for 10sec.) followed by Resin Glaze Liquid & light cure for 3 min.


Coat with Resin Glaze*



04

#CRB needed only in the absence of inhibition layer.
 *Temporary coating with Resin Glaze for up to 6 months.

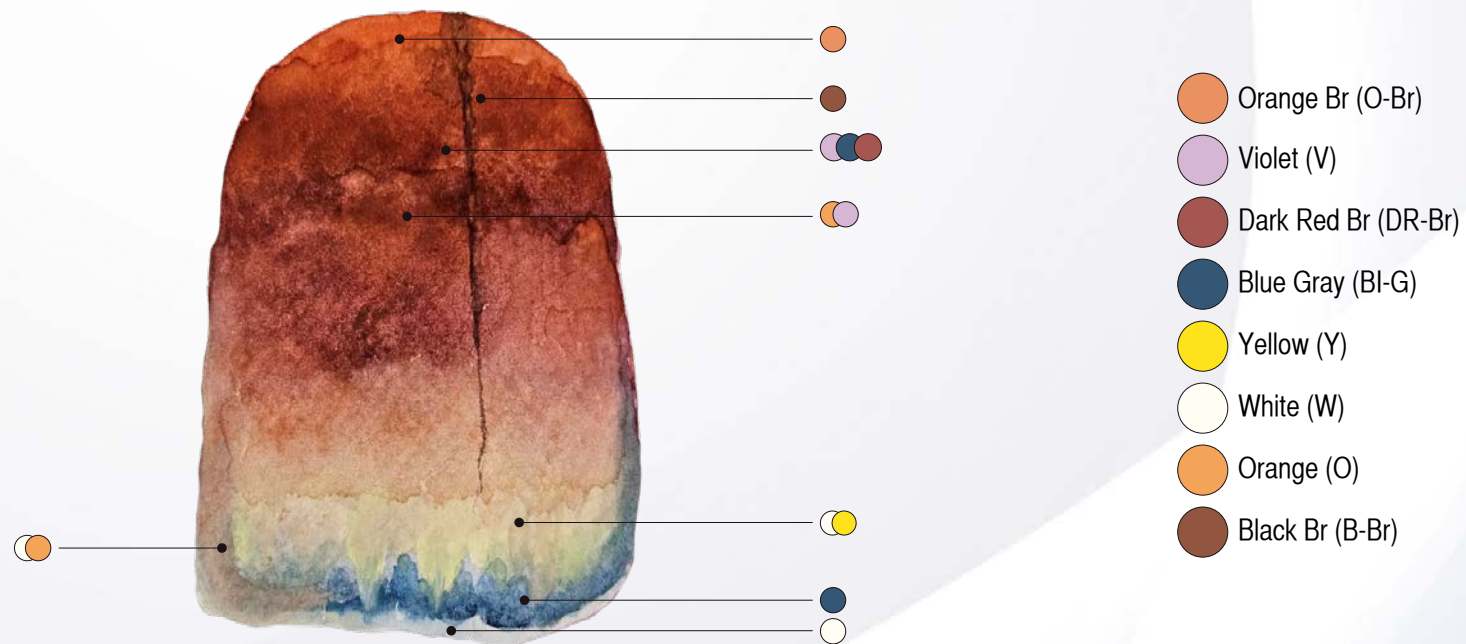
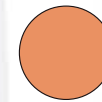
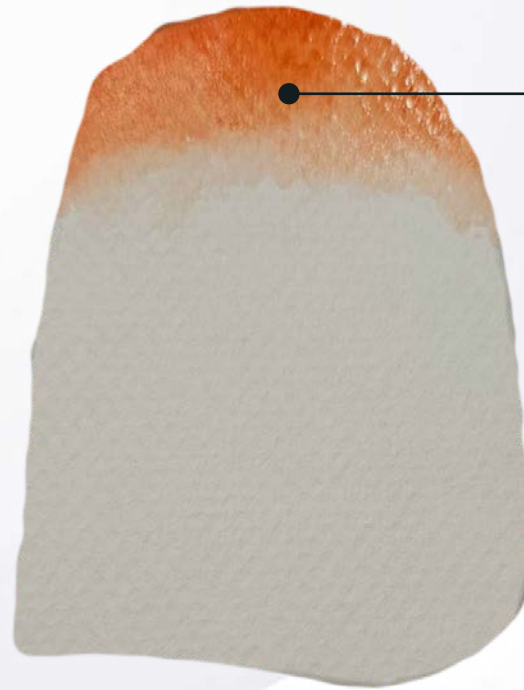
Orange Br (O-Br)

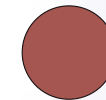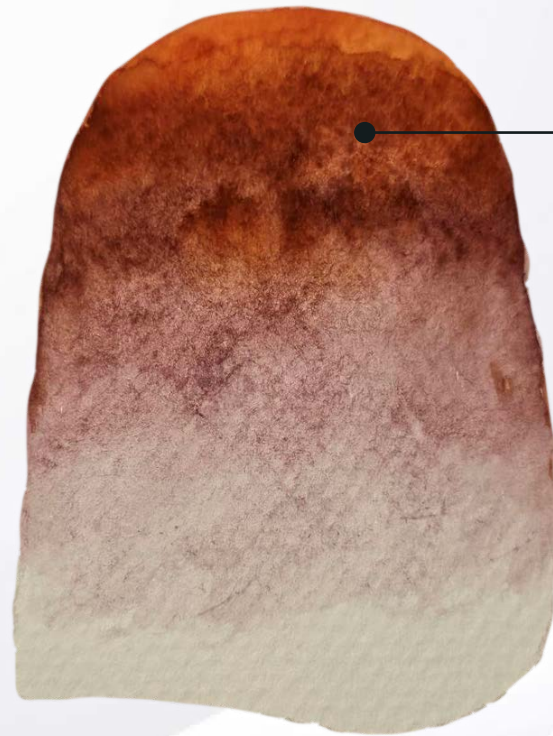
Apply at cervical area

Dark Red Br (DR-Br)

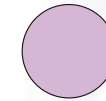Violet (V)


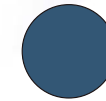
Blue Gray (BI-G)

Apply at cervical area


Blue Gray (BI-G)

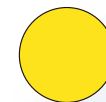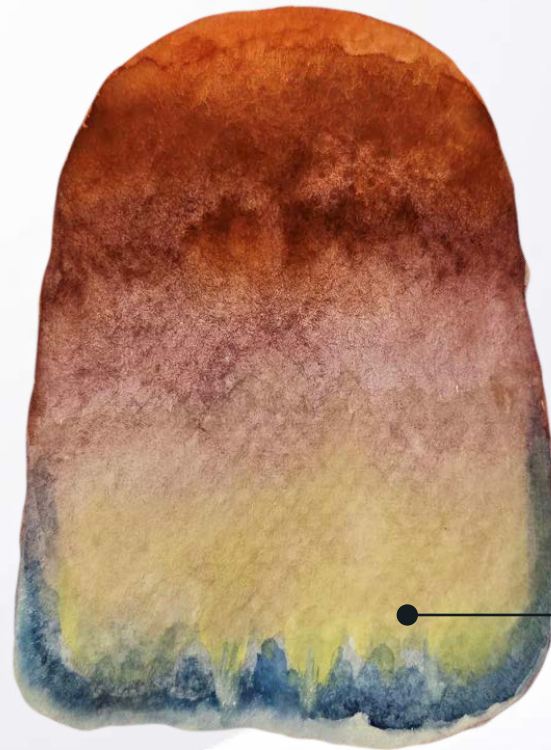Apply at incisal edge & mamelons

Yellow (Y)

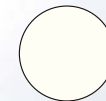White (W)

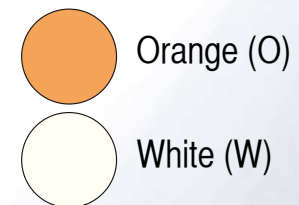
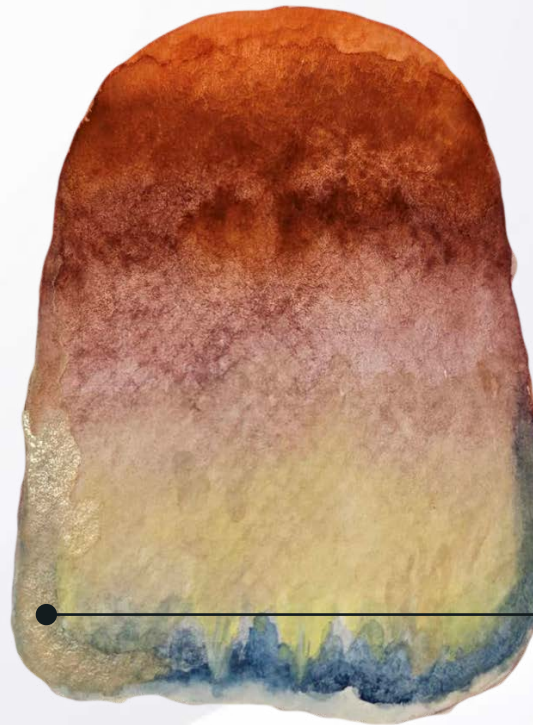
Apply at incisal third



Apply at incisal edge


White (W)

Apply at mesial incisal angle

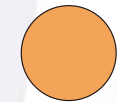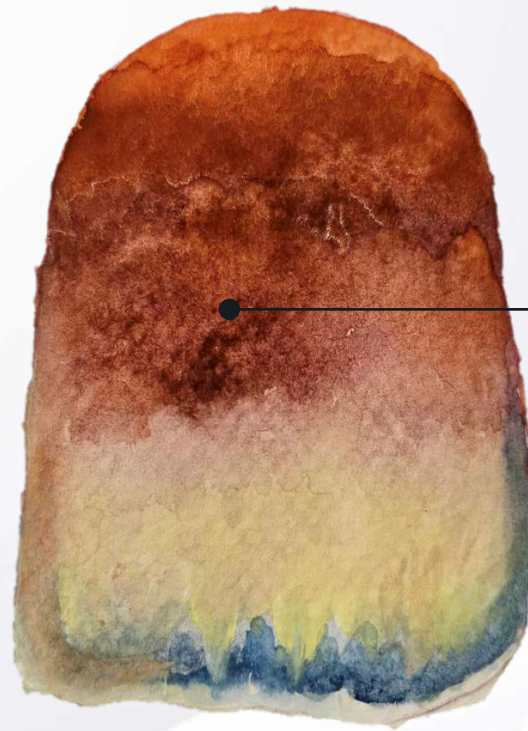
Orange (O)

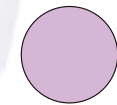Violet (V)

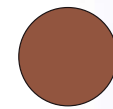Apply at middle third

Black Br (B-Br)

Crack Line Characterization

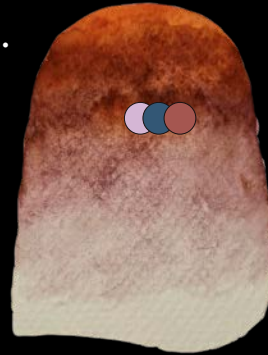
01.


 Orange Br (O-Br)

02.

 Blue Gray (BI-G)
 Violet (V)
 Dark Red Br (DR-Br)

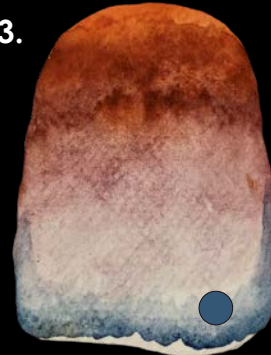03.


 Blue Gray (BI-G)

04.


 Yellow (Y)
 White (W)

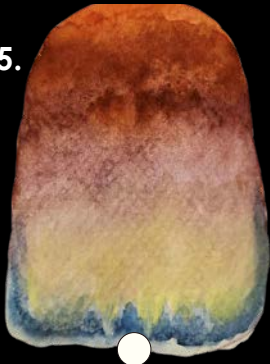05.

 White (W)

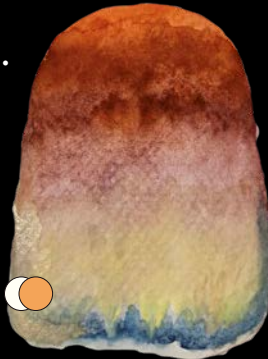06.


 White (W)
 Orange (O)

07.

 Orange (O)
 Violet (V)

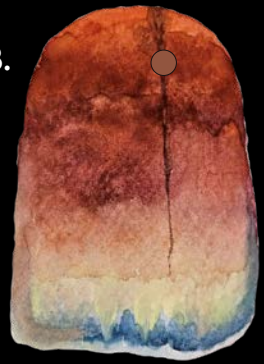08.

 Black Br (B-Br)