ZirGloss

02:2020-0172261

en Instructions for use

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en

READ CAREFULLY BEFORE USE

Polishing Compound for Zirconia, Lithium Disilicate and Porcelain

INDICATIONS FOR USE

Polishing of ceramic restorations including Zirconia, Lithium disilicate and porcelain

PRECAUTIONS AND WARNINGS

- Use protective mask and eyewear when operating handpiece. In case of accidental contact with skin or eyes, rinse with running water.
- Do not use this material intraorally. (This product is intended for extraoral use only.)
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container. (Example ☐ YYYY-MM-DD → Year-Month-Date of the expiration date)
- The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- After contouring and finishing restorations, super-polish them with this material.
- Take adequate amount of this material with a polishing brush or a felt mounted on a dental handpiece (dental electric engine, micro motor etc.).
 - Never exceed the maximum permissible rotation speeds specified in the following table in order to avoid spattering this material
- If the rotary instruments to be used in conjunction with this product withstands lower maximum permissible rotation speed than this product, do not exceed its lower rotation speed.
- When taking this material with polishing brush or felt, use gentle pressure to avoid spattering the material.
- Tightly close the cap after each use.
- After polishing with this material, rinse it off with steam cleaner etc.

(Note)

- There might be some bubbles on the surface of this material but they do not affect the product quality.
- The material might discolor due to moisture or other cause. However, discoloration does not affect the product quality.

COMPOSITION

Diamond, Aluminum oxide, White beeswax, Paraffin and others

STORAGE

Store at room temperature (1-30 °C). Avoid high temperature and high humidity. Keep away from any source of ignition and direct sunlight.

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