# ClearForm X A65

INJECTION TECHNIQUE

- DE Gebrauchsanweisung
- EN Instructions for use
- FR Mode d'emploi
- ES Instrucciones de uso
- IT Istruzioni per l'uso
- PT Manual de instruções

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### Elastomeric bite registration material on vinyl polysiloxane basis, addition-curing

Thank you for the confidence you have shown in our company. You have made the right decision with the purchase of this bite registration material. Please be sure to read these instructions for use through carefully before starting to use the product! Bite registration materials are intended exclusively for use in dental applications by qualified professionals. The medical device must be employed responsibly and with due caution.

### Product description

ClearForm X is manufactured from high-quality raw materials and offers a wide range of applications due to its different final hardnesses. The material features very high thixotropy, is easy to apply thanks to the automatic safety cartridge system, can be easily removed from the mouth, and is permanently dimensionally stable.

### Indication

Bite registration with required visual control; occlusal stamp in light-curing composite restorations; aids and silicone keys for the fabrication (e.g. using the injection method) of veneers and (temporary) crowns and bridges; transfer matrix for the fixation of brackets.

### Bite registration material in 50 ml safety cartridges

Insert the cartridge in the 1:1 mixing gun, remove the end cap and dispense material until material is flowing evenly from both openings. Now place the mixing tip (single-use) over the openings (ensure that the colour code of the end cap and mixing tip are identical) and ensure that it is positioned correctly. To close it, turn the tip 90° clockwise. Operate the mixing gun again and dispense material in the desired amount. The used mixing tip remains on the cartridge and serves as a cap. Prior to the next use, remove the used mixing tip, check the opening for a plug that forms in isolated cases, clean if necessary and proceed as usual.

### Disinfection

The registrations can be disinfected with a 2% glutaraldehyde solution. After removing the impression from the mouth, rinse it under running water for 15 seconds. Then disinfect the impression by immersing it in the disinfectant thoroughly. Observing the disinfectant manufacturer's specifications is essential.

Further information can be found at [www.dgzmk.de](http://www.dgzmk.de), and in J. Prosthet. Dent. 1999 May, 81(5), 621.

### Working and safety instructions

ClearForm X must only be used in accordance with the specified instructions for use. The user has sole liability for any other use which contravenes these instructions for use.

- Bite registration materials are not suitable for conventional impression-taking due to the strength of their final hardness.
- Products marked "single use" are intended for a single use. To avoid cross-contamination, disinfect cartridges and mixing guns before the next use.
- Solutions used prior to the registration, such as mouth rinses or retraction solutions, may interfere with the setting reaction and must be removed thoroughly.
- Contact between the material and certain types of gloves, e.g., latex gloves, should be avoided as the catalyst can be damaged. We recommend nitril / vinyl or PE-based gloves.
- Do not leave any bite registration material residues in the oral cavity.
- Avoid contact with clothing, as cross-linked silicones are chemically resistant and leave stains which cannot be removed.
- Do not use bite registration materials with polyether, polysulphides or condensation-curing silicones.
- To prevent possible incompatibilities, do not combine bite registration materials with A-silicones from other manufacturers.
- Storage: Store in a dry place at between 5 °C and 27 °C (41 °F and 80.6 °F).
- Shelf life: See date printed on package / label. Do not use after expiry date.
- Processing temperature: Room temperature between 18 °C and 25 °C (64.4 °F to 77 °F).

### Technical data and product features

Colour	Transparent
Dosage (1:1)	Automatic
Working time incl. mixing time*	30 sec
Time in mouth	60 sec
Total setting time	90 sec
Hardness	approx. Shore A 65
Linear dimensional change	0.2 %

\* At 23 °C / 73.4 °F. The times specified in the table above are based on 23 °C and a relative humidity of 50 +/- 10 %. As a general rule: High temperatures accelerate setting and lower temperatures delay it.

### Contraindication

The product must not be used in patients with known allergies to flavourings or other ingredients (see safety data sheet or ask the manufacturer). This should be clarified before use!

### Warnings

- **Eye contact:** Avoid contact with eyes. In case of contact, rinse eyes immediately with plenty of clean water and consult an ophthalmologist without delay.
- **Swallowing:** If swallowed, rinse out the mouth and drink plenty of water. Always contact a doctor.
- **Allergic reactions:** It is not possible to rule out allergic reactions in hypersensitive people. If acute allergic reactions should occur during the treatment, treatment with the product should be stopped immediately and a doctor should be consulted

Should a serious incident occur unexpectedly in connection with this medical device, it must be reported to the manufacturer and the competent authority in your Member State.

**Please also observe the warnings in the safety data sheet!**  
Information at [www.shofu.de](http://www.shofu.de)

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