

ClearForm X

INJECTION TECHNIQUE

CLICK / SCAN



FOR MORE INFO

TRANSPARENT SILICONE FOR EASY INJECTION

Clearform X is a transparent polyvinyl siloxane material with ideal thixotropicity, designed for precise, visually guided, effortless injection into all areas, capturing fine details and accurately replicating wax-ups, digital designs, and mock-up models. Optimal clarity ensures efficient light transmission for remarkable photo polymerization of the injectable composite.



Available in a set with 2 cartridges:

ClearForm X Set (REF CFXA65)

2 x 50 ml cartridges
8 mixing tips (green)

CLINICAL APPLICATIONS

- Facilitates injection moulded restoration of anterior veneers, provisional crowns and bridges, occlusal stent (stamp) for posterior restorations, bite registration and transfer matrix for fixation of braces

FEATURES

- ✓ Perfectly balanced with 65 Shore A hardness – **Strong & Stable**
- ✓ Optimal Elasticity & Flexibility – **Easy injection & Effortless removal**
- ✓ Ideal Thixotropicity – **Precise replication** of all anatomical details **without any unregulated slump**
- ✓ Sufficient Tear Strength – **Excellent shape retention without any distortion**
- ✓ Adequate working time with Quick set reaction – **Efficient aesthetic workflow with Consistent outcomes**

DIRECTION FOR USE

1. Insert cartridge in the 1:1 mixing gun, remove the end cap and dispense material until it flows evenly from both openings.
2. Place mixing tip (single-use) over the openings (ensure the colour code of the end cap and mixing tip are identical) and position correctly.
3. Turn the tip 90° clockwise to lock in position and dispense material. Leave mixing tip on as it serves as a cap.
4. Replace with new tip before next use. Clean the opening to remove any plugs at the opening before use.

RELATED PRODUCTS

- Beautiful Injectable X
- Beautiful Injectable XSL
- BeutiBond X-Treme (REF Y2448)

