

ClearForm X

INJECTION TECHNIQUE

Transparent Silicone for Easy Injection

Clear Control – Pure Precision




Ease **Aesthetic** Workflow...

Ensure **Consistent** Outcomes...

Clearform X is a clear polyvinyl siloxane material that ensures precision with visual acuity for injection moulded restoration of highly aesthetic anterior veneers, provisional crowns and bridges, occlusal stent (stamp) for posterior restorations, bite registration and transfer matrix for fixation of braces.

Ideal thixotropicity provides complete control with no slumping and greater dimensional stability with precise flow to reproduce fine details. Optimal clarity of the silicone material enables visual control for quick and easy replication of wax-up and mock-up models and efficient light penetration for higher degree of photopolymerization of the light-cured, injectable composites. The final restoration is quicker to polish and exhibits greater stain resistance, thanks to the absence of residual monomer on the restoration surface.

Simplify Demanding Aesthetic Restorations...

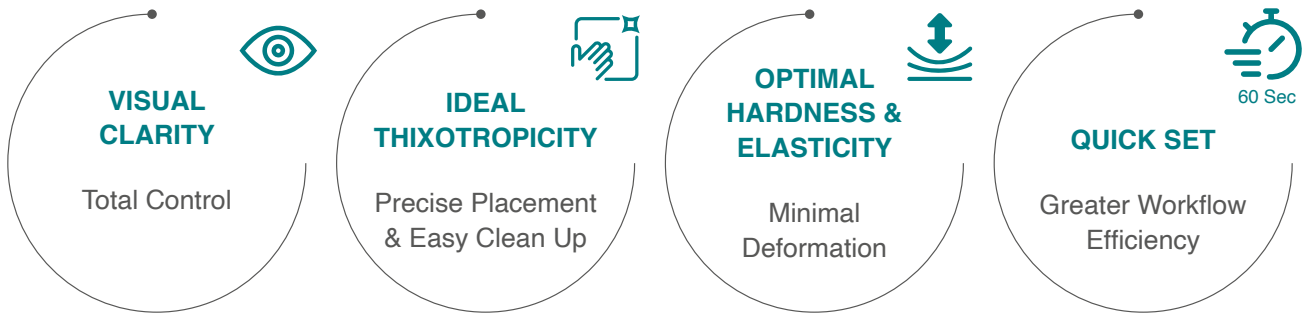

In the Laboratory...

Clearform X silicone index simplifies workflow for transitional and permanent restorations in the lab by enabling a fast and precise fabrication with Ceramage indirect composite.



Benefits

Perfectly Balanced Hardness for Higher Dimensional Stability

Optimal Elasticity for Easy Injection & Removal

Ideal Thixotropicity ensures Precise Application without Unregulated Slump



Tear Resistant & Warp Free

Adequate Working Time with Quick Set Reaction

Well-balanced Rigidity for Accurate Replication of Fine Details

In Clinical Practice...

Clearform X Silicone works synergistically with injectable composites to replicate all the fine details for seamless restorations.

Technique Video



01



Pre-op View: Diastema between 21 & 22

02



Fabricate a 3D printed model from a digital wax-up of tooth #22.

03



Separate # 21 and 23 with PTFE (Teflon) Tape and etch with 35% Phosphoric acid, rinse and scrub with Beautibond Xtreme for 20 sec, air dry & light cure for 10 sec (LED light)

04



Inject composite through injection channels and light cure through silicone index. Finish and adjust fine details with One Gloss, Super-Snap and Polish with Direct Dia and Super Snap Buff disks



Order Information



ClearForm X Set [REF CFXA65]
 2 x 50 ml cartridges
 8 mixing tips (Green)

Related Products



BEAUTIFIL Injectable X/SL



BeautiBond Xtreme Set [REF Y2448]



OneGloss Set [REF 0180]



**Super-Snap X-TREME
 Technique Kit [REF 0508]**



**Super-Snap X-TREME
 Technique Mini-Kit [REF N0509]**



**DirectDIA Paste
 [REF 0558]
 & Super-Snap Buff Disks
 [REF L523/L524]**

**Photos Courtesy of:
 DT Florian Roth, Germany
 Dr. Laura Taetzner, Germany
 Dr. Anand Narvekar, India*



SHOFU DENTAL ASIA-PACIFIC PTE. LTD.
 1 Science Park Road, #01-01 / 02 The Capricorn, Science Park II,
 Singapore 117528 Tel: (65) 6377 2722
 eMail: mailbx@shofu.com.sg www.shofu.com.sg



CF20260331